# beurer

## DELUXE 600

### Upper Arm Blood Pressure Monitor

For Canada only / Solo para Canada / Pour le Canada seulement:  
 Manufacturer / Fabricante / Fabricant :  
 Beurer GmbH, Soflinger Str. 218,  
 89077 Ulm, Germany

Distributed by / Distribuido por / Distribue par :  
 Beurer North America LP  
 1 Oakwood Boulevard, Suite 255  
 Hollywood, FL 33020, USA  
 www.shop-beurer.com

**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT**  
 Keep this manual in a safe location for future reference

## ENGLISH

### 1. IMPORTANT SAFETY NOTES

#### Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

**READ THIS ENTIRE MANUAL, THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.**

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
	Indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	Indicates a hazardous situation which, if not avoided, may result in minor or moderate injury
	Addresses practices not related to personal injury, such as product and/or property damage.
	<b>Isolation of applied parts type BF</b> Galvanically isolated applied part (F stands for "floating"); meets the requirements for leakage currents for type B
	<b>Observe the instructions</b> Read the instructions before starting work and/or operating devices or machines
	<b>Direct current</b> The device is suitable for use with direct current only
	<b>Product information</b> Note on important information
	Temperature limitation
	Humidity limitation
	Serial number
	Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water
	Data transfer via <i>Bluetooth</i> ® low energy technology

#### WARNING

- Do not use the blood pressure monitor on newborns or patients with pre-eclampsia. Consult your doctor before using the blood pressure monitor during pregnancy.
- This device may have difficulty determining the proper blood pressure for pregnant women and for users with irregular heartbeat, diabetes, poor circulation of blood, kidney problems, or for users who have suffered from a stroke.
- The pulse display is not suitable for checking the frequency of heart pacemakers or defibrillators.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device. Supervise children around the device to ensure they do not play with it.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- Make sure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access intravascular therapy, or an arteriovenous (AV) shunt.
- Do not use the device at the same time as other medical electrical devices (ME equipment), This could lead to a malfunction of the device and/or an inaccurate measurement.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions remove the cuff from the arm.

#### CAUTION

- Place the cuff on your upper arm only. Do not place the cuff on other parts of the body.
- Please note that when inflating, the functions of the limb in question may be impaired.
- Do not perform measurements more frequently than necessary. The resulting restriction of the blood flow may cause injury.

#### NOTICE:

- Do not use the device outside of the specified storage and operating conditions. This could lead to incorrect measurements.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.
- Using the blood pressure monitor outside your home environment or whilst on the move (e.g. whilst travelling in a car, ambulance or helicopter, or whilst undertaking physical activity such as playing sport) can influence the measurement accuracy and cause incorrect measurements.
- The blood pressure monitor is made from precision and electronic components. The accuracy of the measured values and service life of the device depend on its careful handling:
  - Protect the device from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
  - Do not touch the device.
  - Do not use the device in the vicinity of strong electromagnetic fields and keep it away from radio systems or mobile telephones.
- Only use the cuff included with the delivery or original replacement cuffs. Otherwise incorrect measured values will be recorded.
- Bring the device to room temperature before taking measurements. If the measuring device has been stored near to the maximum or minimum storage and transport temperature and is brought into an environment with a temperature of 68 °F (20 °C), it is recommended that you wait 2 hours before using the measuring device.
- Never immerse or spill water or any other liquid onto the monitor or any components, otherwise liquid will enter it and cause damage.
- Never attempt to repair, open and/or disassemble the unit (including arm cuff and optional accessory) or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our customer service. Please see warranty for service contact. Before submitting any complaint, first check the batteries and replace them if necessary.
- Only use the device on people who have the specified upper arm measurement for the device.
- Avoid any mechanical restriction, compression or bending of the cuff line.
- To conserve the batteries, the blood pressure monitor switches off automatically if no buttons are pressed for 1 minute.
- Do not expose the device to static electricity. Always ensure that no static electricity is coming off of you before you operate the device.

### 2. INTENDED USE

### 3. PACKAGE CONTENTS


- 1 x Upper Arm Blood Pressure Monitor
- 1 x Universal Cuff (8.7 in - 16.5 in / 22 cm - 42 cm)
- 1 x Storage Pouch
- 4 x 1.5 V AAA batteries
- 1 x Quick Start Guide
- 1 x Instruction manual

### 4. DEVICE DESCRIPTION

This blood pressure monitor performs non-invasive measurement and monitoring of arterial blood pressure and heart rate on adult patients.


### 5. PREPARING FOR MEASUREMENT

- Universal Cuff
- Air Tube Connector
- User Button (1) (2)
- START/STOP** Button (3)
- Bluetooth*® Button (4)
- Connection for Air Tube connector
- Air Tube
- LCD Display
- $\frac{1}{2}$  Buttons
- MEMORY** Button (5)
- Risk Indicator
- Battery Compartment Cover

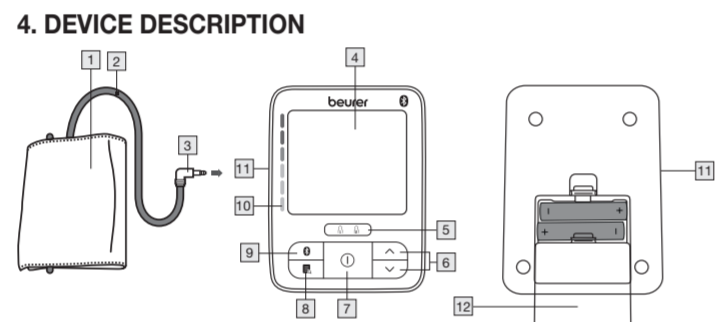1. Universal Cuff
2. Air Tube
3. Air Tube Connector
4. LCD Display
5. User Button (1) (2)
6.  $\frac{1}{2}$  Buttons
7. **START/STOP** Button (3)
8. **MEMORY** Button (5)
9. *Bluetooth*® Button (4)
10. Risk Indicator
11. Connection for Air Tube connector
12. Battery Compartment Cover

#### LCD Display Information
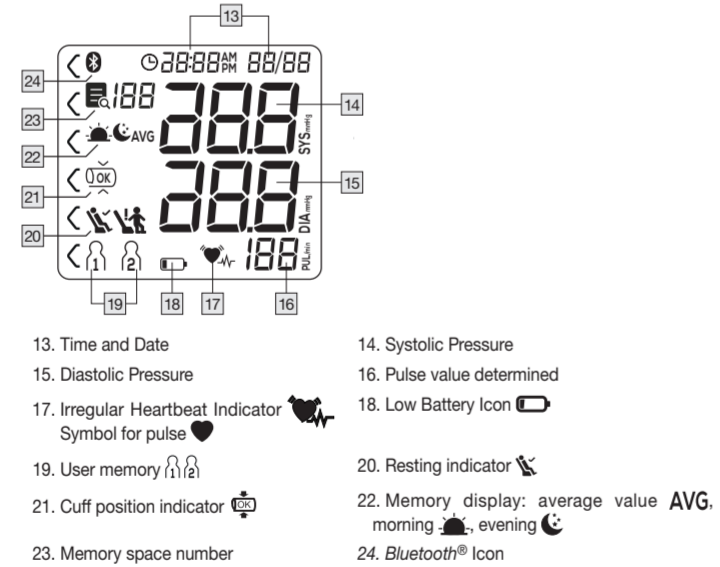
### 5. PREPARING FOR MEASUREMENT

#### Inserting/changing batteries:

- Remove the Battery Compartment Cover from the rear of the device.
- Insert four "AAA" alkaline batteries (included) according to the polarity markings inside the compartment. **NOTE:** Do not use rechargeable batteries.
- Reattach the Battery Compartment Cover.
- All LCD Display elements will briefly appear and "12h" will flash. Set the date and time as described below. Any saved measurements will be retained.

If the battery replacement symbol is permanently displayed, you can no longer perform any measurements and must replace all batteries. Once the batteries have been removed from the device, the date and time must be set again. Any saved measurements are retained.

#### Setting the hour format, date, and time:

If date and time are not set, stored measurements will not have a valid timestamp associated with them. The settings menu can be accessed in two different ways:

#### Before first use and after every battery change:

When you insert batteries into the device, you will automatically arrive at the relevant menu.

#### If batteries have already been inserted:

- With the device switched-on press and hold the *Bluetooth*® Button for 3 seconds.
  - The time format will flash. Select an hour format (12- or 24-hour display) with the  $\frac{1}{2}$  Buttons and press the *Bluetooth*® Button to confirm.
  - The year flashes on the LCD Display. Set the year with the  $\frac{1}{2}$  Buttons and press the *Bluetooth*® Button to confirm.

- The month flashes on the LCD Display. Set the month with the  $\frac{1}{2}$  Buttons and press the *Bluetooth*® Button to confirm.
- The day flashes on the LCD Display. Set the day with the  $\frac{1}{2}$  Buttons and press the *Bluetooth*® Button to confirm.
 

**NOTE:** If the hour format is set to 24h, the day/month display sequence is reversed.
- The hour flashes on the LCD Display. Set the hour with the  $\frac{1}{2}$  Buttons and press the *Bluetooth*® Button to confirm.
- The minutes flash on the LCD Display. Set the minutes with the  $\frac{1}{2}$  Buttons and press the *Bluetooth*® Button to confirm.
- The *Bluetooth*® icon flashes on the LCD Display. Activate ("On" flashes) or deactivate ("Off" flashes) *Bluetooth*® data transfer with the  $\frac{1}{2}$  Buttons and press the *Bluetooth*® Button to confirm. Note that operation with *Bluetooth*® activated will increase battery drain.

#### *Bluetooth*® Setup

In addition to displaying and saving measurements locally on the blood pressure monitor itself, you can also transfer measurements to your mobile device via *Bluetooth*®. To do so, you will need to install the "Beurer HealthManager Pro" app. The app is freely available in either the Apple App Store or Google Play Store.

List of system requirements and compatible devices:

To transfer the measured values, proceed as follows:

- Activate *Bluetooth*® in the smartphone settings.
- Start the app.
- Select DELUXE 600 in the app and follow the instructions.

### 6. MEASURING BLOOD PRESSURE

The blood pressure monitor is suitable for home use and needs to be at room temperature when measuring. Measurements can be performed on either arm.

#### Positioning the Cuff:

- Place the cuff on a bare upper arm. Ensure blood circulation is not restricted by tight clothing or other objects. The cuff must be placed on the upper arm so that the bottom edge is positioned 0.8 in - 1.2 in above the elbow and over the artery, with the Air Tube in line with the center of the palm.
- Fasten the cuff, making sure it is not too tight around your arm. You should be able to fit two fingers under the cuff after it is in place. The cuff is suitable for you if the index mark is within the OK range on the cuff when it is in place.


- Insert the Air Tube Connector into the Air Tube Port.

Blood pressure can vary between the right and left arm, which may mean that the measured blood pressure values are different. Always perform the measurement on the same arm. If the values between the two arms are significantly different, please consult your doctor to determine which arm should be used for the measurement.

**Cuff Position Indicator**  
 This device features a cuff position indicator to check the correct positioning of the cuff on your upper arm during measurement.

When the cuff is correctly positioned, the symbol will appear in the LCD Display during measurement and will also display along with measurement results. If the cuff is positioned incorrectly, the measurement is cancelled and an error message appears in the LCD Display. If this happens, refer to the "Troubleshooting Guide" section for solutions.

#### Correct Posture:

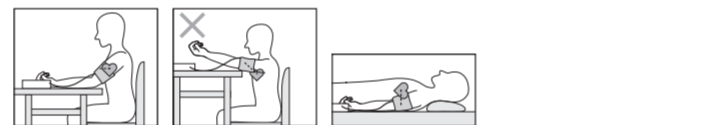- For consistency, always measure your blood pressure at the same time each day.
- Rest for at least five minutes before each measurement.
- Perform the measurement either sitting or lying down. Always make sure that the cuff is level with your heart.
- To perform a measurement, sit comfortably with your arms and back supported. Do not cross your legs and place your feet flat on the floor.
- Keep still during the measurement and do not talk.
- Relax your arm and the palm of your hand.
- When performing multiple measurements on an individual, wait at least 5 minutes between each measurement.
- Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising.
- Repeat the measurement if you are unsure of the results.
- Measurements are for your information only – they are no substitute for a medical examination. Discuss the measurements with your doctor and never base any medical decisions on them (e.g. medicines and their administration).

#### Preparing a Measurement

- There are 2 user memories available, each with 90 memory spaces, so that you can save the measured values of 2 different people separately. To select a desired user memory, press the **START/STOP** Button (3) and then press the **User Button** until the desired user memory icon appears in the LCD Display. Confirm by pressing the **START/STOP** Button again; the device will then automatically begin measuring by inflating the cuff.
- The measurement is taken while the cuff is being inflated. As soon as a pulse has been detected, the pulse symbol will flash in the rhythm of your pulse. After no pulse is detected anymore, the cuff air pressure will be released.
- The measurements for systolic pressure, diastolic pressure and pulse will be displayed. You can stop the measurement at any time by pressing the **START/STOP** button (3).
- The measurement will be saved automatically.
- "Er\_" will appear if the measurement was not performed properly. Refer to the "Troubleshooting Guide" section for details about specific error message numbers and then repeat the measurement.
- If you have activated *Bluetooth*® data transfer, the data will be transferred. The blood pressure monitor displays the symbol for *Bluetooth*® transfer during transfer. To power down press the **START/STOP** button (3).

**Note:** The device switches off automatically after one minute if no button has been pressed.

Wait for at least five minutes before taking another measurement!

### 7. EVALUATING RESULTS

#### Irregular Heartbeat:

This device can detect irregular heartbeat patterns during measurement and will indicate them by displaying the Irregular Heartbeat Icon . This could be an indicator for arrhythmia, a condition where the heart rhythm is abnormal. These symptoms (omitted or premature heartbeats, slow or excessively fast heart rate) may be caused, among other things, by heart disease, age, physical predisposition, excessive use of stimulants, stress, or lack of sleep. This function is not designed for diagnosing or treating an arrhythmic disorder. Only your doctor can determine if you do have arrhythmia.

Repeat the measurement if the Irregular Heartbeat Icon displays after a measurement. Rest for at least five minutes between measurements and do not talk or move during measurement. If the icon appears often, contact your doctor, since self-diagnosis and treatment based solely on the test results may be dangerous. It is vital to follow your doctor's instructions.

#### Resting indicator (using HSD diagnostics)

One of the most common errors made when taking a blood pressure measurement is not ensuring that the user's circulatory system is sufficiently at rest when taking the measurement. In this case, the measured systolic and diastolic blood pressure values do not represent the blood pressure at rest. However, it is this blood pressure at rest that should be used to assess the measured values. This blood pressure monitor uses integrated haemodynamic stability diagnostics (HSD) to measure the user's haemodynamic stability when taking the blood pressure measurement. This enables it to indicate whether the blood pressure was taken when the user's circulatory system was sufficiently at rest.

	The measured blood pressure value was obtained when the user's circulatory system was sufficiently at rest and reliably represents the blood pressure at rest.
	Indicates that the value was obtained when the user's circulatory system was not sufficiently at rest. The blood pressure values measured in this case generally do not represent the blood pressure at rest. The measurement should therefore be repeated after a period of physical and mental rest lasting at least 5 minutes.
No resting indicator symbol is displayed	During the measurement it was not possible to determine whether the user's circulatory system was sufficiently at rest. In this case too, the measurement should be repeated after a period of rest lasting at least 5 minutes.

The user's circulatory system not being sufficiently at rest can be the result of various factors, such as physical stress, mental strain/distraction, speaking or experiencing cardiac arrhythmia during the measurement. In an overwhelming number of cases, HSD will give a very good guide as to whether the user's circulatory system is rested when a blood pressure measurement is taken. However, certain patients suffering from cardiac arrhythmia or chronic mental conditions may remain haemodynamically unstable even in the long-term, something which persists even after repeated periods of rest. The accuracy of the results for the blood pressure at rest is reduced in these users. Like any medical measurement method, the precision of HSD is limited and it can lead to incorrect results in some cases. Nevertheless, the blood pressure measurements taken when the user's circulatory system is sufficiently at rest represent particularly reliable results.

### Risk Classification

Measurements can be classified and assessed by the table below. However, these standard values are only a general guideline, since individual blood pressure varies in different people and different age groups, etc.

Hypertension Category	Systolic (in mmHg)	Diastolic (in mmHg)	LED Risk Indicator
Grade 3: Severe hypertension	≥ 180	≥ 110	Red
Grade 2: Moderate hypertension	160–179	100–109	Orange
Grade 1: Mild hypertension	140–159	90–99	Yellow
High Normal	130–139	85–89	Green
Normal	120–129	80–84	Green
Optimal	< 120	< 80	Green

Source: WHO, 1999

It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The classification on the display and the scale on the unit show which category the recorded blood pressure values fall into. If the systolic and diastolic values fall into two different categories, the Risk Classification Indicator on the device always shows the higher category.

### 8. DISPLAYING AND DELETING MEASUREMENTS

#### User Memory:

Every successful measurement is stored along with its date and time. If there are more than 90 measurements in a particular user memory bank, the oldest measurement is deleted.

#### Average Values:

With the device switched off, press the **START/STOP** button (3), select the desired user memory with the **User Button**.

- Confirm your selection by pressing the **memory** button.
- "AVG" will flash in the display and an average of all the saved measurements in the selected user memory will be shown.
- Press the  $\frac{1}{2}$  button again to display the average value of the last 7 days for the morning measurement. (Morning: 5.00 am-9.00 am, display ). Press the  $\frac{1}{2}$  button again to display the average value of the last 7 days for the evening measurement. (Evening: 6 pm-8 pm, display ). By pressing the  $\frac{1}{2}$  button again, the last individual measurement will be displayed with the date and time. Switch through all the saved measurements by pressing the  $\frac{1}{2}$  button again.
- Press the **START/STOP** button (3) for 2 seconds to switch off the device.

#### Clearing Memory:

- To delete the memory of the selected user, press the **memory** button when the device is switched on.
- The average value of all measurements will appear on the display. Next to this, "AVG" and the *Bluetooth*® transfer symbol will flash, if *Bluetooth*® function is turned on.
- Keep the function buttons  $\frac{1}{2}$  pressed simultaneously for 5 seconds.
- After successful deletion, "CL 0\_" will appear in the display.

The data has been deleted.

#### Restoring the device to its factory settings

You can delete all of the data saved on the device, e.g. settings or saved measured values, by restoring the device to its factory settings.

- With the device switched on, press the **memory** button. You will then be in the most recently selected user memory. "AVG" will flash in the display.
- Now press the  $\frac{1}{2}$  button again to switch to the display mode for the average morning value. will flash in the display.
- Hold down both function buttons  $\frac{1}{2}$  at the same time for 15 seconds, until "CL" appears in the display. Now all of the data saved on the device will be deleted and the device will be restored to its factory settings.

### 9. CARE AND MAINTENANCE

- Clean the device and cuff with a slightly moistened cloth.
- Do not use detergents or solvents.
- Never immerse the device in water or other liquids.
- When storing the device, make sure no heavy objects are placed on top of it. Remove the batteries. Ensure the Cuff line does not have any sharp kinks.

#### Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

### 10. TROUBLESHOOTING GUIDE

In case of errors, Er\_ will appear in the LCD Display, where \_ is a number 1-7. Error messages may appear if:

Trouble shooting	Problem	Solution
"Er1"	The pulse cannot be measured correctly.	Repeat the measurement process. Be sure not to move or talk.
"Er2"	You have been talking or moving during the measurement process.	
"Er3"	The cuff was attached too tightly or too loosely.	
"Er4"	An error occurred during the measurement process.	
"Er5"	The pump-up pressure is higher than 300mmHg or the blood pressure measurement falls outside of the measuring range.	Repeat the measurement process. Be sure not to move or talk.
"Er6"	A device error has occurred.	Repeat the measurement process. Be sure not to move or talk.
"Er7"	Data cannot be sent via <i>Bluetooth</i> ®.	Activate <i>Bluetooth</i> ® on your smartphone and start the "Beurer HealthManager Pro" app.
+ "bL0"	The batteries are almost dead.	Reinsert the batteries or replace them with new batteries.

### 11. TECHNICAL SPECIFICATIONS

Type	DELUXE 600
Model	BM 82
Measurement Method	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measurement Range	Measuring range 30-280 mmHg Systolic 50 – 280 mmHg Diastolic 30 – 200 mmHg Cuff pressure 0 – 300 mmHg Pulse 40 – 200 beats per minute

Display Accuracy	Systolic ± 3 mmHg, Diastolic ± 3 mmHg, Pulse ± 5% of the displayed value
Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: systolic 8 mmHg/diastolic 8 mmHg
Memory	2 x 90 memory spaces
Dimensions	L 4.92 in x W 3.74 in x H 1.65 in (L 125 mm x W 95 mm x H 42 mm)
Weight	0.77lb (350g) with cuff, without batteries
Cuff Size	8.7 in - 16.5 in (22 - 42 cm)
Operating Range	50°F - 104°F (10 °C - 40 °C), <90% relative humidity (non-condensing), 800-1050 hPa ambient pressure
Storage Range	-4 °F - 131 °F (-20 °C - 55 °C), ≤ 90% relative humidity (non-condensing), 800-1050 hPa ambient pressure
Power Supply	4 x 1.5V  AAA batteries
Battery Life	Approx. 350 measurements

The serial number is located on the device or in the battery compartment. Technical information is subject to change without notification to allow for updates.

- This unit is in line with IEC 80601-2-30 Safety and Performance, IEC 60601-1 Electrical Safety and IEC 60601-1-2 Electromagnetic Compatibility.
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life.

### 12. FCC COMPLIANCE INFORMATION

Upper Arm Blood Pressure Monitor BM 82

Responsible Party – U.S. Contact Information

Beurer North America LP  
 1 Oakwood Boulevard, Suite 255  
 Hollywood, FL 33020  
 United States  
 1-800-536-0366  
 info@beurer.com

#### FCC Compliance Statement

This device complies with Part 15 of the FCC Rules.Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### 13. ICES COMPLIANCE INFORMATION

This device contains licence-exempt transmitter(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s).

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