# beurer

# SI 30

**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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**Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.**

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## CONTENT

|                                 |   |   |   |
|---------------------------------|---|---|---|
| 1. Important safety notes ..... | 2 | 5. Care, Maintenance and Disposal ..... | 5 |
| 2. Package Contents .....       | 4 | 6. Technical Specifications .....       | 6 |
| 3. Parts and Controls.....      | 4 | 7. Warranty .....                       | 6 |
| 4. To Use .....                 | 4 |   |   |

## SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

**If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact customer service. See warranty for service contact.**

### Dear Valued Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for application in the areas of heating, weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Register your product today to receive further benefits:

- added protection for your purchase
- health and wellness tips
- new product information
- continual outstanding customer service and support.

Visit [www.registerbeurer.com](http://www.registerbeurer.com) or contact customer service at **1-800-536-0366**. Please read these instructions for use carefully and keep them for later use. Make them accessible to other users and observe the information they contain.

With kind regards,  
Your Beurer team

## 1. Important safety notes

### Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

**READ THIS ENTIRE MANUAL, THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.**

|   |  |
|---|--|
|                      | This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death. |
| <br><b>WARNING</b>   | Indicates a hazardous situation which, if not avoided, could result in death or serious injury.  |
| <br><b>CAUTION</b> | Indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.   |
| <br><b>NOTICE</b>  | Addresses practices not related to personal injury, such as product and/or property damage.  |

|   |   |
|---|---|
|   | <b>Product information</b><br>Note on important information   |
|  | <b>Protection class II device</b><br>The device is double-protected and thus corresponds to protection class 2. |

**WARNING – To reduce the risk of fire, electric shock, or serious personal injury:**

- The appliance should always be placed on a firm, flat waterproof surface. Be sure the appliance is in a stable position and the power cord is out of the way to prevent the unit from being tipped over.
- Protect the device from heavy impact.
- The appliance should not be left unattended. Keep out of reach of children.
- Keep packaging material away from children (risk of suffocation).
- Do not use the appliance while bathing.
- Do not immerse in water or any other liquid. Do not overflow.
- Never move the appliance while in use. It can spill hot water if tilted or upset causing injury.
- Do not use appliance while sleepy or drowsy.
- Do not use in area where aerosol products are being used or where oxygen is being administered.
- Before using the appliance, extend the cord and inspect for any signs of damaged. Do not use the unit if the cord has been damaged.
- The appliance has a polarized plug (one blade is wider than the other) as a safety feature.
- This plug will fit into a polarized outlet only one way. If the plug does not fit fully into the outlet reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature.

- To avoid fire or shock hazard, plug the appliance directly into a 120 VAC electrical outlet.
- The appliance should always be unplugged and emptied when not in operation or while being cleaned. Shut off and unplug the appliance before moving. Do not move or tilt while in operation. Plug and unplug unit with dry hands. Never pull by the cord to unplug.
- Do not operate the appliance without water. Turn off and unplug when the unit is empty.
- To avoid the risk of fire, shock, or personal injury, do not use an extension cord.
- Appliance requires regular cleaning. Refer to and follow cleaning instructions.
- Use this appliance only as described in this manual and only with attachments or accessories recommended.
- Never add any medications or inhalant liquids to the Water Reservoir.
- Do not operate outdoors. This appliance is intended for indoor use.
- Do not cover or insert objects into any openings in the unit.
- Do not attempt to repair or adjust any electrical or mechanical functions on this appliance. Doing so will void your warranty.
- For residential use only.

**Notes on electromagnetic compatibility**

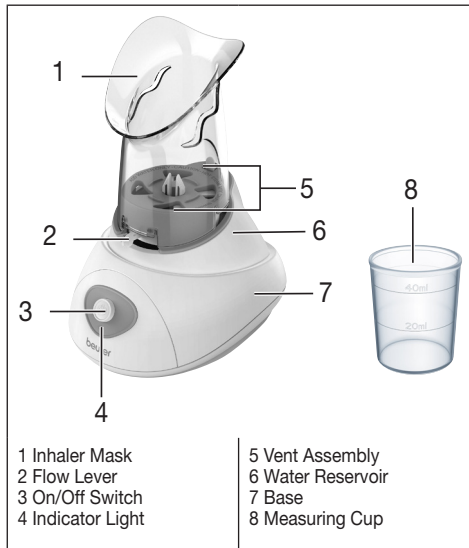
- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in faulty operation.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.

- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

## 2. Package Contents

- 1 x Steam Inhaler SI 30
- 1 x Mask
- 1 x Measuring Cup
- 1 x Instruction manual

## 3. Parts and Controls

## 4. To Use

1. Fully extend the power cord. Failure to do so could cause overheating and possible fire hazard. Do not use with an extension cord. Place Inhaler on a flat, stable, water-resistant surface for use.

2. Remove the Inhaler Mask.

3. Press the buckle and lift the Vent Assembly straight up to separate it from the Base.

4. Fill the Water Reservoir with clean, clear water up to the MAX mark (1.35 fl oz /40 ml). Do not overfill. Use with water only. Do not add any medication or inhalant liquids to the Water Reservoir.

5. Replace the Vent Assembly. Be sure the slots align with the location hole on the top of the Base and then press the buckle to connect the Vent Assembly to the Base.

6. Replace Inhaler Mask.
7. Connect the power cord to an electrical outlet. Do not use with an extension cord. Press the On/Off Switch; the Indicator Light will illuminate in green.
8. After 3-5 minutes, the Steam Inhaler will begin to emit steam.

**CAUTION:** The water reservoir and water will become very hot.

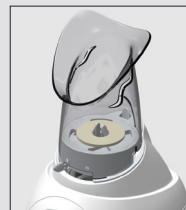
9. Slide the Flow Lever left to decrease the amount of steam and to the right to increase the amount of steam. Place your face into the Inhaler Mask; this may cause steam to escape under the mask or cause the inhaler to tip over. Breathe normally through your nose and mouth, inhaling steam into your nasal and throat passages. Continue treatment for five to fifteen minutes (see next section for how to Regulate Steam Flow and Temperature).

**WARNING:** Never move or lift the inhaler while it is in use. Steam is produced by boiling water; tilting can cause water spill and possible burns.

10. When treatment is completed, press On/Off switch to switch off the device; the Indicator Light will shut off. Disconnect the power cord from the electrical outlet. Let the Steam Inhaler cool before moving or refilling it (see the Care and Maintenance section). Do not operate the Steam Inhaler when it is empty.

## How to Regulate Steam Flow and Temperature

The Steam Inhaler features an adjustable Vent Assembly which allows you to use the Flow Lever to control the amount of steam you inhale as well as regulate temperature.

## Using Inhalant Pads

This Steam Inhaler features an inhalant pad holder. Ensure that the product is powered OFF before inserting an Inhalant Pad.

1. You can use a menthol inhalant pad with your steam inhaler to provide up to eight hours of soothing menthol vapors.
2. Open the Inhalant Pad by tearing a notch on the pad bag. Do not touch the pad with your hands. If the pad contacts your hands/fingers, do not rub your face or eyes as it may cause irritation.
3. Place the Inhalant Pad on top of the Vent Assembly. The center hole of the pad should sit around the tabs.
4. Discard pad after use.

## 5. Care, Maintenance and Disposal

**CAUTION:** Always disconnect the power cord from the electrical outlet and let the steam inhaler cool down for at least 20 minutes before cleaning.

### After Each Use

1. Remove Inhaler Mask and Vent Assembly and wipe dry with a clean cloth or paper towel.

- If any water is left in the Water Reservoir, pour the water out and wipe the Water Reservoir and Base with a dry, clean cloth or paper towel. Never immerse the Base in water or other liquid.
- Replace the Vent Assembly and Inhaler Mask. Store in a cool, dry place.

## Routine Maintenance

Perform the following cleaning procedures if using the Steam Inhaler ten or more times a week, when scale builds up in the Water Reservoir, or if not used for long periods.

- Remove Inhaler Mask and Vent Assembly.
- Pour two ounces of undiluted white vinegar into the Water Reservoir and let sit for 10 minutes.
- Empty the white vinegar and clean the Water Reservoir with fresh, clean tap water.
- Dry the Water Reservoir and Base with a clean cloth or paper towel.
- Wash Inhaler Mask with mild soap and warm water then carefully rinse off all soap with fresh, clean tap water.
- Dry the Inhaler Mask with a clean cloth or paper towel.
- Replace Vent Assembly and Inhaler Mask. Store them in a cool, dry place.

## Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

## 6. Technical Specifications

|                        |   |
|------------------------|---|
| Model:                 | SI 30   |
| Dimensions (L x W x H) | 6 x 7.6 x 9.7 inches (153.5 x 193 x 247.5 mm) |
| Power supply:          | 120 V / 60 Hz                                 |

|  |  |
|--|--|
| Current consumption:                     | 510-650 mA (steady use), 670-830 mA (first minute) |
| Power consumption:                       | 70 W (steady use), 90 W (first minute)             |
| Weight:                                  | approx. 19.2 oz (544 g) (with mask and water cup)  |
| Filling volume of internal water heater: | 1.35 fl oz (40 ml)                                 |
| Operating Conditions:                    | 50°F~104°F (10°C~40°C), 30~85% RH                  |
| Storage and transportation conditions:   | 14°F~122°F (-10°C~50°C), 30~85% RH                 |

## 7. Warranty

Your Beurer Steam Inhaler, model SI 30, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Steam Inhaler, model SI 30, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at [info@beurer.com](mailto:info@beurer.com) to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with

proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR**

**LIABILITES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY.**

Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

**IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.** Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit: [www.beurer.com](http://www.beurer.com)

Distributed by:  
Beurer North America LP  
1 Oakwood Boulevard, Suite 255  
Hollywood, FL 33020, USA  
[www.beurer.com](http://www.beurer.com)

Questions or comments? Call our US-based  
customer service toll free at 1-800-536-0366.

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