beurer

SI 30

READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT

Keep this manual in a safe location for future reference

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**Questions or comments? Call our US-based customer service
toll free at 1-800-536-0366.**

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CONTENT

1. Important safety notes	2	5. Care, Maintenance and Disposal	5
2. Package Contents	3	6. Technical Specifications	6
3. Parts and Controls.....	4	7. Warranty	6
4. To Use	4		

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact customer service. See warranty for service contact.

1. Important safety notes

Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

READ THIS ENTIRE MANUAL, THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
---	--

 WARNING	Indicates a hazardous situation which, if not avoided, could result in death or serious injury.
 CAUTION	Indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
 NOTICE	Addresses practices not related to personal injury, such as product and/or property damage.
	Product information Note on important information
	Protection class II device The device is double-protected and thus corresponds to protection class 2.

WARNING

- The appliance should always be placed on a firm, flat waterproof surface. Be sure the appliance is in a stable position and the power cord is out of the way to prevent the unit from being tipped over.
- Protect the device from heavy impact.
- The appliance should not be left unattended. Keep out of reach of children.
- Keep packaging material away from children (risk of suffocation).
- Do not use the appliance while bathing.

- Do not immerse in water or any other liquid. Do not overfill.
- Never move the appliance while in use. It can spill hot water if tilted or upset causing injury.
- Do not use appliance while sleepy or drowsy.
- Do not use in area where aerosol products are being used or where oxygen is being administered.
- Before using the appliance, extend the cord and inspect for any signs of damaged. Do not use the unit if the cord has been damaged.
- The appliance has a polarized plug (one blade is wider than the other) as a safety feature.
- This plug will fit into a polarized outlet only one way. If the plug does not fit fully into the outlet reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature.
- To avoid fire or shock hazard, plug the appliance directly into a 120 VAC electrical outlet.
- The appliance should always be unplugged and emptied when not in operation or while being cleaned. Shut off and unplug the appliance before moving. Do not move or tilt while in operation. Plug and unplug unit with dry hands. Never pull by the cord to unplug.
- Do not operate the appliance without water. Turn off and unplug when the unit is empty.
- To avoid the risk of fire, shock, or personal injury, do not use an extension cord.
- Appliance requires regular cleaning. Refer to and follow cleaning instructions.
- Use this appliance only as described in this manual and only with attachments or accessories recommended.
- Never add any medications or inhalant liquids to the Water Reservoir.
- Do not operate outdoors. This appliance is intended for indoor use.
- Do not cover or insert objects into any openings in the unit.

- Do not attempt to repair or adjust any electrical or mechanical functions on this appliance. Doing so will void your warranty.
- For residential use only.

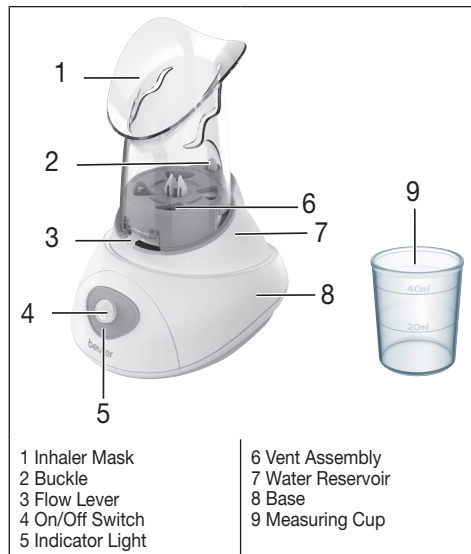
Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in faulty operation.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

2. Package Contents

- 1 x Steam Inhaler SI 30
- 1 x Mask
- 1 x Measuring Cup
- 1 x Instruction manual

3. Parts and Controls

4. To Use

1. Fully extend the power cord. Failure to do so could cause overheating and possible fire hazard. Do not use with an extension cord. Place Inhaler on a flat, stable, water-resistant surface for use.

2. Remove the Inhaler Mask.

3. Press the buckle and lift the Vent Assembly straight up to separate it from the Base.

4. Fill the Water Reservoir with clean, clear water up to the MAX mark (1.35 fl oz / 40 ml). Do not overfill. Use with water only. Do not add any medication or inhalant liquids to the Water Reservoir.

5. Replace the Vent Assembly. Be sure the slots align with the location hole on the top of the Base and then press the buckle to connect the Vent Assembly to the Base.

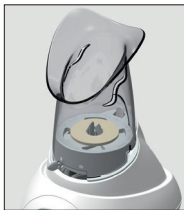
6. Replace Inhaler Mask.

7. Connect the power cord to an electrical outlet. Do not use with an extension cord. Press the On/Off Switch; the Indicator Light will illuminate in green.
8. After 3-5 minutes, the Steam Inhaler will begin to emit steam.
CAUTION: The water reservoir and water will become very hot.
9. Slide the Flow Lever left to decrease the amount of steam and to the right to increase the amount of steam. Place your face into the mask. Do not rest your face on the Inhaler Mask; this may cause steam to escape under the mask or cause the inhaler to tip over. Breathe normally through your nose and mouth, inhaling steam into your nasal and throat passages. Continue treatment for five to fifteen minutes (see next section for how to Regulate Steam Flow and Temperature).
WARNING: Never move or lift the inhaler while it is in use. Steam is produced by boiling water; tilting can cause water spill and possible burns.
10. When treatment is completed, press On/Off switch to switch off the device; the Indicator Light will shut off. Disconnect the power cord from the electrical outlet. Let the Steam Inhaler cool before moving or refilling it (see the Care and Maintenance section). Do not operate the Steam Inhaler when it is empty.

How to Regulate Steam Flow and Temperature

The Steam Inhaler features an adjustable Vent Assembly which allows you to use the Flow Lever to control the amount of steam you inhale as well as regulate temperature.

Using Inhalant Pads

This Steam Inhaler features an inhalant pad holder. Ensure that the product is powered OFF before inserting an Inhalant Pad.

1. You can use a menthol inhalant pad with your steam inhaler to provide up to eight hours of soothing menthol vapors.
2. Open the Inhalant Pad by tearing a notch on the pad bag. Do not touch the pad with your hands. If the pad contacts your hands/fingers, do not rub your face or eyes as it may cause irritation.
3. Place the Inhalant Pad on top of the Vent Assembly. The center hole of the pad should sit around the tabs.
4. Discard pad after use.

5. Care, Maintenance and Disposal

CAUTION: Always disconnect the power cord from the electrical outlet and let the steam inhaler cool down for at least 20 minutes before cleaning.

After Each Use

1. Remove Inhaler Mask and Vent Assembly and wipe dry with a clean cloth or paper towel.
2. If any water is left in the Water Reservoir, pour the water out and wipe the Water Reservoir and Base with a dry, clean cloth or paper towel. Never immerse the Base in water or other liquid.
3. Replace the Vent Assembly and Inhaler Mask. Store in a cool, dry place.

Routine Maintenance

Perform the following cleaning procedures if using the Steam Inhaler ten or more times a week, when scale builds up in the Water Reservoir, or if not used for long periods.

1. Remove Inhaler Mask and Vent Assembly.
2. Pour two ounces of undiluted white vinegar into the Water Reservoir and let sit for 10 minutes.
3. Empty the white vinegar and clean the Water Reservoir with fresh, clean tap water.
4. Dry the Water Reservoir and Base with a clean cloth or paper towel.
5. Wash Inhaler Mask with mild soap and warm water then carefully rinse off all soap with fresh, clean tap water.
6. Dry the Inhaler Mask with a clean cloth or paper towel.
7. Replace Vent Assembly and Inhaler Mask. Store them in a cool, dry place.

Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

6. Technical Specifications

Model:	SI 30
Dimensions (L x W x H)	6 x 7.6 x 9.7 inches (153.5 x 193 x 247.5 mm)
Power supply:	120 V / 60 Hz
Current consumption:	510-650 mA (steady use), 670-830 mA (first minute)
Power consumption:	70 W (steady use), 90 W (first minute)
Weight:	approx. 19.2 oz (544 g) (with mask and water cup)
Filling volume of internal water heater:	1.35 fl oz (40 ml)
Operating Conditions:	50°F~104°F (10°C~40°C), 30~85% RH

Storage and
transportation
conditions:

14°F~122°F (-10°C~50°C),
30~85% RH

7. Warranty

5-Year Limited Warranty for Original Purchaser

Your Beurer Steam Inhaler, Model SI30, is warranted to be free from defects in materials and workmanship for a period of five (5) years from the date of purchase under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Steam Inhaler, Model SI30, without additional charge, for any part or parts covered by this written warranty. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service, contact our customer service department at 1-800-536-0366 or info@beurer.com with a description of the problem. If the issue is deemed to fall within the scope of the 5-Year Limited Warranty, you will be asked to mail the product (at your cost) in its original package along with proof of purchase, your name, address, and phone number. If the issue is not deemed eligible under warranty, we will provide a quotation for repair or replacement, including return shipping fees.

Third-Party Repairs

This warranty complies with the federal Magnuson-Moss Warranty Act. You may use third-party parts or services for repairs without voiding this warranty, unless the repair itself causes damage to the product. We encourage you to retain your proof of purchase

and contact our customer service team with any questions.

Unauthorized Sellers

This warranty applies only to products purchased from Beurer or authorized retailers and distributors. Products purchased from unauthorized sellers — including third-party platforms or online marketplaces (such as eBay or unverified Amazon sellers) — are not covered by this warranty. This policy is in place to protect customers from counterfeit, altered, or used products that may not meet Beurer's quality standards.

If you're unsure whether your seller is authorized, please contact our support team.

Exclusions

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessories; alterations; improper installation; misapplication; lack of reasonable care; unauthorized repairs or modifications; improper electrical/power supply; worn batteries; normal wear and tear; power loss; dropped product; failure to follow instructions; lack of recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions.

This warranty is void if the product is used in a commercial or business environment.

Beurer's maximum liability under this warranty is limited to the actual purchase price paid, as confirmed by proof of purchase. The warranty is effective only in the country where the product was purchased and used.

Legal Notice

This warranty gives you specific legal rights, and you may have other rights which vary from state to state. THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO MERCHANTABILITY OR FITNESS

FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE LENGTH OF THIS WRITTEN WARRANTY.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING FROM USE OR INABILITY TO USE THE PRODUCT.

Beurer does not authorize any third party — including retailers or resellers — to modify the terms of this warranty.

This warranty does not apply to opened, used, repackaged, or resold products, including those sold through online auctions or unauthorized channels.

For more information regarding our product line in the USA, please visit: www.shop-beurer.com

Distributed by:

Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020, USA
www.shop-beurer.com

Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.

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