beurer

BR 90

- EN** Insect bite healer
Instructions for use.....3
- ES** Curador de picaduras de insectos
Instrucciones de uso31
- FR** Appareil soulageant les piqûres
d'insectes
Mode d'emploi62

For Canada only/ Solo para Canadá/

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Manufacturer / Fabricante / Fabricant :

Beurer GmbH, Söflinger Str. 218,
89077 Ulm, Germany

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www.shop-beurer.com

Assembled in China / Ensamblado en
China / Assemblé en Chine

**Questions or Comments? Call our
US-based customer service toll free at
1-800-536-0366.**

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**READ THIS MANUAL COMPLETELY AND CAREFULLY
BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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ENGLISH

Contents

1. Important safety notes	5	7. Care, Maintenance and Disposal	20
2. Getting to Know your Instrument ..	14	8. Troubleshooting Guide	21
3. Package Contents	15	9. Technical Specifications	23
4. Parts and Controls.....	16	10. FCC Compliance Information.....	24
5. To Set Up.....	17	11. Warranty	26
6. To Use	17		

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact customer service. See warranty for service contact.

Dear Valued Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for application in the areas of heating, weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Register your product today to receive further benefits:

- added protection for your purchase
- health and wellness tips
- new product information
- continual outstanding customer service and support.

Visit www.registerbeurer.com or contact customer service at 1-800-536-0366.

Please read these instructions for use carefully and keep them for later use. Make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer team

1. Important safety notes

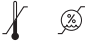
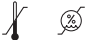
Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

READ THIS ENTIRE MANUAL, THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
	Product information Note on important information

WARNING	Indicates a potentially hazardous situation, which, if not avoided, could result in death or serious injury.
CAUTION	Indicates a potentially hazardous situation, which, if not avoided, may result in minor/moderate injury.
NOTICE	Addresses practices not related to personal injury, such as product and/or property damage.
	Follow instructions for use
	Type BF applied part
	On
○	Off
	Light

	Serial number
IP 42	IP class Device protected against solid foreign objects ≥ 1 mm and against water dripping at an angle
Storage/Transport 	Permissible storage and transport temperature and humidity
Operating 	Permissible operating temperature and humidity
	With optional light

 **WARNING**

Before using the device, read the safety notes carefully and follow the instructions exactly whenever using the device. Non-observance of the following instructions can result in injury or material damage. Store the instructions for use for future reference and make them accessible to other users. Any person using it on themselves must be over 12 years old. The product should only be used on children under 12 years old if supervised or assisted by an adult.

- This device may be used by persons with limited physical, sensory or mental capabilities, provided they are supervised by a responsible person or they receive detailed instructions on how to use the device.
- Do not use the device if you suffer from diabetes. Sensitivity to pain could, in certain circumstances, be reduced and diabetics may burn themselves.
- Do not use the device on people with disabilities, children under three years old or people with reduced sensitivity to heat. This includes people with skin alterations due to illness or scarred tissue in the application area, after taking pain relief medication, drinking alcohol, or taking drugs.

- Do not use the device on acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin, whether painful or not, reddened skin, rashes, (e.g. allergies), burns, frostbite, bruises, swellings, both open and healing wounds, and post-operative scars where the healing process could be affected).
- Do not use the device if you are suffering from a fever.
- Do not use the device on sensitive areas of skin.
- Do not use the device if you have any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- Do not use the device if you are using lotions, creams, and gels locally at the same time.
- Do not use the device if you suffer from persistently irritated skin due to long-term heat application on the same area of skin.
- Before using the device, consult your doctor if you have any skin conditions, serious illness (in particular if you have a propensity to thromboembolic conditions or recurrent malignant growths), or unexplained chronic pain in any part of the body.

- Seek medical advice immediately if the symptoms persist or worsen despite using the insect bite healer. Stop using the device immediately.
- Seek medical advice immediately for a tick bite. The bite may result in transmission of pathogens such as tick-borne encephalitis (TBE) or borreliosis.
- Seek medical advice immediately at the first signs of increased itching, skin redness, swelling, dizziness, nausea, shortness of breath, formation of welts, or a marked change in blood pressure.
- Seek medical advice immediately if the symptoms worsen whilst or after using the insect bite healer and stop using it.
- Do not use near the eyes, on the eyelids, on the mouth, or on mucous membranes.
- Do not allow children to play with the device.
- Store the device out of the reach of children and pets.
- Keep packaging material away from children (risk of suffocation).
- The device is not a substitute for medical consultation and treatment. Always consult your doctor if you are experiencing any pain or are suffering from an illness.

- If the device does not work properly, or if you feel unwell or experience pain, stop using it immediately.

CAUTION

- Do not touch the device with wet hands. Do not allow water to spray onto the device. Only operate the device if it is completely dry.
- Do not use any detergents or solvents to clean and maintain the device. If liquid penetrates the device this could cause damage to the electronics or other device parts and lead to a malfunction.
- The device should only be used by one person; use by multiple people is not recommended.
- Protect the device against direct sunlight and store in a dry place.
- Protect the device from impacts, moisture, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).
- Ensure that no water enters the device. If this should happen, remove the batteries and only use the device again when it is completely dry.
- If you store the device, no heavy objects should be placed on top of it.

- Before use, ensure that there is no visible damage to the device or accessories. In case of any doubt, do not use the device, and contact your retailer or the specified Customer Service address.
- Store the device in a location protected against climatic influences. The device must be stored in the environmental conditions specified.
- Do not leave the device unattended while in use. If the device malfunctions, switch it off and stop using it.
- No user-serviceable parts inside; do not repair or adjust the device yourself. Opening the device case will void the warranty.

Battery Handling Safety Precautions

- Use only the size and type of batteries specified.
- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.

- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions.)
- Do not dispose of batteries in fire, batteries may explode or leak.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.

Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.

- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

2. Getting to Know your Instrument


This product is intended solely for household use in relieving itching or swelling caused by insect bites or stings through the use of heat. Do not use this product for any other purpose.

Many insect bites and stings result in painful and swollen bite/sting marks which can often cause severe itching. This insect bite healer offers effective relief for insect bites and stings through a targeted application of heat. As a result, the itching subsides and your symptoms improve; it is often noted that the area heals more comfortably.

3. Package Contents

- 1 x insect bite healer BR 90
- 2 x batteries, type AAA (LR03 1.5 V)
- 1 x instruction manual

4. Parts and Controls

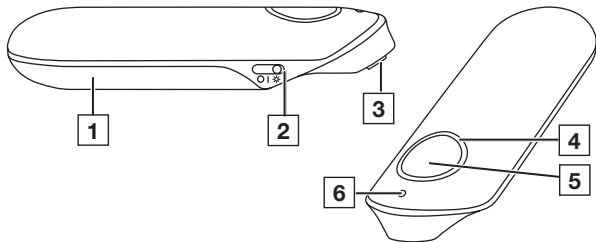1 Battery Compartment Cover

2 Power/Light Switch

3 Heating Plate/Optional light

4 Ring Light

5 Activation Button

6 Status LED

5. To Set Up

1. Slide the Battery Compartment open and insert two “AAA” size alkaline batteries (included) into the Battery Compartment in accordance with the polarity markings inside the compartment. Reattach the Battery Compartment Cover by sliding it closed until it clicks into place.
2. Before beginning the treatment, make sure the device is clean, dry, and intact.

6. To Use

Make sure the insect’s stinger is no longer lodged in the skin. If needed, remove the stinger carefully. Using this device immediately after the insect bite/sting yields the best results. Waiting too long to treat the bite/sting will enable itching and swelling to start and the insect bite healer will only have a limited effect. Even if treatment cannot begin right away, do so as soon as possible, as this will usually still alleviate some itching and speed up the healing process.

CAUTION: Depending on the treated person’s skin sensitivity, the heat from the device may feel unpleasant and cause slight skin redness and, in rare cases,

irritated skin. If the heat function feels too hot, stop treatment immediately.

1. Slide the Power/Light Switch to the “I” (On) position. The Status LED will illuminate in blue, which indicates the device is ready to use.
2. To activate the Light, slide the Power/Light Switch another notch forward toward the Light icon (☀️).
3. Place the Heating Plate of the device directly onto the area that has been stung or bitten. Two application durations are available, depending on the skin sensitivity of the person being treated:
 - 3-second treatment: for the first use or for use on sensitive skin. Press and release the Activation Button once. Half of the Ring Light will illuminate and the treatment will start.
 - 6-second treatment: for regular use on normal skin. Press and release the Activation Button twice quickly. The Ring Light will illuminate fully and the treatment will start.

To shut the device off at any time, press and hold the Activation Button for about two seconds. In order to use the device again, slide the Power/Light

Switch to the “O” (Off) position and then to the “I” (On) position to switch the device back on.

4. After the selected treatment duration, the device will beep and the Heating Plate and Ring Light will shut off. Remove the Heating Plate from the skin.
 5. If needed, you can reapply the heat function of the insect bite healer to the same sting or bite after a 2-minute break, or you can immediately use it on another sting or bite in a different area. However, please ensure that you do not exceed a maximum of 5 applications per hour on the same treatment area.
 6. To shut the device off (including the Light, if activated), slide the Power/Light Switch to the “O” (Off) position.
-  If the device is accidentally left in the “I” (On) position and to prevent unintentional activation while inside a pocket or purse, the device will automatically switch off 2 minutes after use. In order to use the device again, slide the Power/Light Switch to the “O” (Off) position and then to the “I” (On) position to switch the device back on.

7. Care, Maintenance and Disposal

Battery Levels

When the battery level becomes low, the Status LED will flash and the device will emit three short beeps. Replace both batteries as described in the “To Set Up” section.

Cleaning and Storage

- Ensure the Heating Plate has cooled down before storing.
- Shut the device off and remove the batteries before cleaning.
- Clean the heating plate after each use with an alcohol-based pad/wipe. Also clean the device between uses on multiple areas of the same person.
- Keep the device dry. Never place, store, or use this device near bathtubs, showers, sinks, or other containers of water.
- Never use any substances for cleaning that could potentially be toxic if contact the skin or mucous membranes.
- Do not use abrasive cleaning products or mechanical cleaning devices such as brushes on this device.
- If not using the device for a long period of time, clean it and store it in a dry

location out of direct sunlight.

Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

8. Troubleshooting Guide

Problem	Solution
If the device does not turn on	Ensure the power switch is in the ON position.
	Make sure the batteries are not depleted.
If the device does not emit a sufficient heat pulse	Battery level is low; replace batteries.
Device beeps three times and Status LED flashes	Battery level is low; replace batteries.

Problem	Solution
Device is on and batteries replaced, but no heat pulse is emitted during treatment	Device has turned off automatically. Either the activation button was pressed for more than 2 seconds or it was not used for more than 2 minutes and the device turned off automatically. Slide the Power/Light Switch to the "O" (Off) position and then to the "I" (On) position to switch the device back on.
Device beeps five times and Ring Light flashes	Device may be damaged or faulty; Shut device off and wait 10 minutes before restarting. If the device does not begin working normally upon restart, contact Customer Service

9. Technical Specifications

Type	BR 60
Model	BR 90
Max. temperature	122 °F ± 3.6 °F (50 °C ± 2 °C)
Operating Duration per Battery Set, without light	Approx. 300 applications
Power supply	2 x 1.5 V AAA LR03 batteries
Dimensions (L x W x H)	3.98 x 1.4 x 0.77 in (101 x 35.5 x 19.5 mm)
Weight (with batteries)	1.65 oz (47 g)
Functions	2 application times (3s/6s)
Operating Ambient Temperature Range	50 - 104° F (10 - 40° C)
Storage Temperature Range	14 - 122° F (-10 - 50° C)
Operating and Storage Humidity Range	30 - 85%
Automatic device switch-off	2 min after last application

The serial number is located on the device or in the battery compartment.
Subject to technical changes.

10. FCC Compliance Information

Insect Bite Healer BR 90

Responsible Party U.S. Contact Information:

Beurer North America LP

1 Oakwood Boulevard, Suite 255

Hollywood, FL 33020

United States

1-800-536-0366

info@beurer.com

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

11. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer Insect Bite Healer, model BR 90, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Insect Bite Healer, model BR 90, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not

deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For more information regarding our product line in the USA, please visit:
www.beurer.com

For Canada only:
Manufacturer:
Beurer GmbH
Söflinger Str. 218
89077 Ulm Germany

Distributed by:
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020, USA
www.shop-beurer.com

Questions or comments? Call our US-based customer service toll free at
1-800-536-0366.

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