beurer

BR 110

EN	Insect bite healer	
	Instructions for use	3
ES	Aliviador de picaduras de insectos	
	Instrucciones de uso	28
FR	Guérisseur de piqures d'insectes	
	Mode d'emploi	55

For Canada only / Solo para Canadá /

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Manufacturer / Fabricante / Fabricant :

Beurer GmbH, Söflinger Str. 218,
89077 Ulm, Germany

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Assembled in China / Ensamblado en China /

Assemblé en Chine

**Questions or Comments? Call our
US-based customer service toll free at
1-800-536-0366.**

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**READ THIS MANUAL COMPLETELY AND CAREFULLY
BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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ENGLISH

Contents

1. IMPORTANT SAFETY NOTES.....	4	7. Care, Maintenance and Disposal ...	16
2. What to know about your device ...	11	8. Troubleshooting Guide.....	18
3. What's included.....	11	9. Technical Specifications	20
4. Device Overview.....	12	10. FCC Compliance Information.....	22
5. Initial use	13	11. Warranty	24
6. Usage	14		

Read these instructions for use carefully. Observe the warnings and safety notes. Keep these instructions for use for future reference. Make the instructions for use accessible to other users. If the device is passed on, provide the instructions for use to the next user as well.

1. IMPORTANT SAFETY NOTES

Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
	Product information Note on important information
 WARNING	Indicates a potentially hazardous situation, which, if not avoided, could result in death or serious injury.
 CAUTION	Indicates a potentially hazardous situation, which, if not avoided, may result in minor/moderate injury.
NOTICE	Addresses practices not related to personal injury, such as product and/or property damage.

	Observe the instructions Read the instructions before starting work and/or operating devices or machines
	Type BF applied part
	Serial number
	Device protected against solid foreign objects ≥ 1 mm in diameter and larger, and against drops of water when the housing is angled up to 15°
Storage/Transport 	Permissible storage and transport temperature and humidity
Operating 	Permissible operating temperature and humidity

 WARNING

Before using the device, read the safety notes carefully and follow the instructions exactly whenever using the device. Non-observance of the following instructions can result in injury or material damage. Store the instructions for use for future reference and make them accessible to other users. Any person using it on themselves must be over 12 years old. The product should only be used on children under 12 years old if supervised or assisted by an adult.

- Do not use the device if you suffer from diabetes. Sensitivity to pain could, in certain circumstances, be reduced and diabetics may burn themselves.
- Do not use the device on people with disabilities, children under three years old or people with reduced sensitivity to heat. This includes people with skin alterations due to illness or scarred tissue in the application area, after taking pain relief medication, drinking alcohol, or taking drugs.
- Do not use the device on acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin, whether painful or not, reddened skin, rashes, (e.g. allergies), burns, frostbite, bruises, swellings, both open and healing wounds, and post-operative scars where the healing process could be affected).
- Do not use the device if you are suffering from a fever.

- Do not use the device on sensitive areas of skin.
- Do not use the device if you have any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- Do not use the device if you are using lotions, creams, and gels locally at the same time.
- Do not use the device if you suffer from persistently irritated skin due to long-term heat application on the same area of skin.
- Before using the device, consult your doctor if you have any skin conditions, serious illness (in particular if you have a propensity to thromboembolic conditions or recurrent malignant growths), or unexplained chronic pain in any part of the body.
- Seek medical advice immediately if the symptoms persist or worsen despite using the insect bite healer. Stop using the device immediately.
- Seek medical advice immediately for a tick bite. The bite may result in transmission of pathogens such as tick-borne encephalitis (TBE) or borreliosis.
- Seek medical advice immediately at the first signs of increased itching, skin redness, swelling, dizziness, nausea, shortness of breath, formation of welts, or a marked change in blood pressure.

- Seek medical advice immediately if the symptoms worsen whilst or after using the insect bite healer and stop using it.
- Do not use near the eyes, on the eyelids, on the mouth, or on mucous membranes.
- Do not allow children to play with the device.
- Store the device out of the reach of children and pets.
- The device is not a substitute for medical consultation and treatment.
 Always consult your doctor if you are experiencing any pain or are suffering from an illness.
- If the device does not work properly, or if you feel unwell or experience pain, stop using it immediately.
- **Choking hazard!** Keep the device, the packaging and the accessories away from children, as there is a risk of suffocation, injury and choking. Seek medical attention immediately if swallowed.
- Please note the general hygiene measures when using the insect bite healer.

CAUTION

- Do not touch the device with wet hands. Do not allow water to spray onto the device. Only operate the device if it is completely dry.
- The device should only be used by one person; use by multiple people is not recommended.

- Depending on the treated person's skin sensitivity, the heat from the device may feel unpleasant and cause slight skin redness and, in rare cases, irritated skin. If the heat function feels too hot, stop treatment immediately.
- Protect the device from impacts, moisture, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).
- Ensure that no water enters the device. If this should happen, remove the device and only use the device again when it is completely dry.
- If you store the device, no heavy objects should be placed on top of it.
- Do not leave the device unattended while in use.
- If the device malfunctions, stop using it.

NOTICE

- Do not use any detergents or solvents to clean and maintain the device. If liquid penetrates the device this could cause damage to the electronics or other device parts and lead to a malfunction.
- Protect the device against direct sunlight and store in a dry place.
- Before use, ensure that there is no visible damage to the device or accessories. In case of any doubt, do not use the device, and contact your retailer or the specified Customer Service address.

- Store the device in a location protected against climatic influences. The device must be stored in the environmental conditions specified.
- No user-serviceable parts inside; do not repair or adjust the device yourself. Opening the device case will void the warranty.

Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

2. What to know about your device

Intended Use

This product is intended solely for household use in relieving itching or swelling caused by insect bites or stings through the use of heat. Do not use this product for any other purpose.

How it works

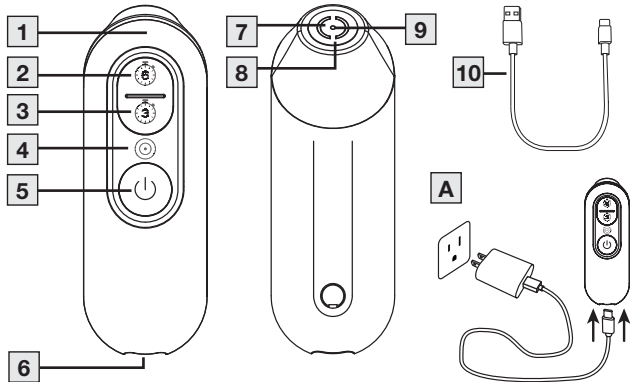
Many insect bites and stings result in painful and swollen bite / sting marks which can often cause severe itching. Remove any insect stinger before use. For best result, treat affected area immediately. Later treatment may still help reduce itching and speed healing but will be less effective.

The insect bite healer applies heat to the affected area which relieves itching and discomfort. It is often noted that the treated area heals more comfortably.

3. What's included

- 1x BiteX Max BR110
- 1x quick start guide
- 1x instruction manual

4. Device Overview

- | | |
|--|---|
| 1 Device status LED | 2 Activation button
(6 sec. program) |
| 3 Activation button
(3 sec. program) | 4 LED to indicate the selected
hot-plate size |
| 5 ON/OFF button | 6 USB-C port |
| 7 Inner hot plate | 8 Optional second heating ring |
| 9 Spotlight LED | 10 USB-C charging cable |

5. Initial use

Charge the device **A**

1. Connect the USB-C charging cable **10** provided to a USB mains adapter (output: max. 5 V/1 A, not included in delivery) and the insect bite healer to the USB-C port **6**.
2. Then insert the mains adapter into a suitable socket.

3. Whilst charging, the device status LED **1** flashes blue. Once the device is fully charged, the device status LED **1** lights up blue continuously.

6. Usage

1. To switch on the device, press and hold the ON/OFF button **5** for 2 seconds
→ Device status LED **1** starts to light up green. Device is now ready for use.
2. You can choose between two hot-plate sizes depending on which treatment area you prefer.
 - The inner hot plate **7** is preset as the default size.
→ The inner LED to indicate the selected hot-plate size **4** lights up.
 - If you prefer a particularly large treatment area, briefly press the ON/OFF button **5** once to activate the outer heating ring **8**, in addition to the standard size.
→ The outer LED to indicate the selected hot-plate size **4** also lights up.
 - Each additional short press of the ON/OFF button **5** switches between the normal and large treatment area.
3. Place the device with its hot plate **7** **8** on the area that has been stung or bitten.

4. You can choose between 2 different treatment times (3 or 6 seconds) depending on how sensitive your skin is.

The 3-second treatment is recommended for first use or use on sensitive skin.

A) 3-second treatment:

Briefly press the 3sec-activation button **3** once.

→ The device status LED **1** lights up orange and the treatment starts.

A) 6-second treatment:

Briefly press the 6sec-activation button **2** once.

→ The device status LED **1** lights up pink and the treatment starts.

5. An acoustic signal sounds at the end of the treatment.

→ The device status LED **1** lights up green again and the hot plate **7** **8** stops providing heat.

6. To prolong the battery life, switch off the device each time after using it. To switch off the device, press and hold the ON/OFF button **5** for 2 seconds.

If you do not actively switch off the device yourself, the automatic switch-off function will switch it off automatically after 2 minutes. To switch off the device, press and hold the ON/OFF button **5** for 2 seconds.

Battery warning

Before the battery level becomes too low for other applications, the device status LED **1** starts to flash 3x and an acoustic signal sounds 3x. Charge the device promptly as described in chapter 5.

7. Care, Maintenance and Disposal

Care and Maintenance

Observe the following hygiene instructions to avoid health risks:

- The device is intended for multiple uses.
- It is recommended that the device is cleaned before first use and each additional use.
- The device must be switched off and allowed to cool down each time before cleaning.
- Use only soft, dry cloths for cleaning the housing.

- Never use any substances for cleaning that could potentially be toxic if they came into contact with the skin or mucous membranes, or if they were swallowed or inhaled. Do not use any abrasive cleaning products and never submerge the device in water.
- Do not clean the device in the dishwasher!
- Avoid mechanical cleaning with brushes or similar, as this could result in irreparable damage.
- When using the device on multiple areas that have been stung and bitten, it is recommended that you clean the heating plate each time. This reduces the possibility of germs and bacteria being transferred from one area of skin to the next. To disinfect the heating plate, use a cotton bud moistened with disinfectant.

Disposal

Follow local regulations for disposal. If you have any questions please contact the local authorities responsible for waste disposal.

8. Troubleshooting Guide

Problem	Possible cause	Solution
The device does not switch on.	The device's battery is empty.	Charge the device as described in chapter 5.
The device no longer gets hot enough.	The battery level is too low.	Charge the device as described in chapter 5.
The device outputs 3 acoustic signals and the device status LED 1 flashes 3 times – even after switching the device OFF and ON again.	Battery level is too low, meaning that the device may no longer be able to be operated.	Charge the device as described in chapter 5.

Problem	Possible cause	Solution
The device switches off each time the treatment is started using the activation buttons 2 3 and the hot plate does not heat up.	The emergency shutdown has been activated. The treatment button 2 3 was pressed for longer than 2 seconds.	To switch the device on again, press and hold the ON/OFF button 5 for 2 seconds again. Briefly press the relevant activation button 2 3 once to start the treatments.
The device does not switch on even though it has been charged. The device status LED 1 does not light up either when charging or when switching on the device.	The device may be faulty.	Please contact Customer Services.

Problem	Possible cause	Solution
The device outputs 5 acoustic signals and the device status LED 1 flashes 5 times.	This indicates a potential malfunction.	Switch the device off and repeat the application again after 10 minutes. Please contact Customer Services if the fault occurs again.

9. Technical Specifications

Type	BR 110
Dimensions (L x W x H)	3.89 x 1.38 x 0.83 in (98.8 x 35.0 x 21.0 mm)
Weight	Device: approx. 1.41 oz (40g) incl. charging cable: approx. 1.76 oz (50g)
Internal Power supply	rechargeable Li-polymer battery, 3.7 V, 600mAh Charge: 5V $\overline{\overline{=}}$ 1A; 5W

External power supply (adapter not included)	Input: 100–240 V $\sim$ 50/60Hz Output: 5 V $\equiv$ 1A
Max. temperature	122 °F $\pm$ 3.6 °F (50 °C $\pm$ 2 °C)
Functions	2 time settings (3s/6s) 2 hot-plate sizes (normal (0.28 in/7 mm) / large (0.55 in/14 mm))
Expected operating duration	Approx. 500 applications (per battery charge)
Storage and transport temperature and humidity	-4°F to 131°F (-20°C to 55°C), relative humidity 0% to 90%
Operating temperature and humidity	41°F to 104°F (5°C to 40°C), relative humidity 15% to 90%, 700–1060 hPa ambient pressure
Automatic device switch-off	2 min after last application

The serial number is located on the device.
Subject to technical changes.

10. FCC Compliance Information

Insect Bite Healer BR 110

Responsible Party U.S. Contact Information:

Beurer North America LP

1 Oakwood Boulevard, Suite 255

Hollywood, FL 33020

United States

1-800-536-0366

info@beurer.com

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

11. Warranty

5-Year Limited Warranty for Original Purchaser

Your Beurer Insect Bite Healer, Model BR 110, is warranted to be free from defects in materials and workmanship for a period of five (5) years from the date of purchase under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners. We will, at our option, repair or replace the Beurer Insect Bite Healer, Model BR 110, without additional charge, for any part or parts covered by this written warranty. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service, contact our customer service department at 1-800-536-0366 or info@beurer.com with a description of the problem. If the issue is deemed to fall within the scope of the 5-Year Limited Warranty, you will be asked to mail the product (at your cost) in its original package along with proof of purchase, your name, address, and phone number. If the issue is not deemed eligible under warranty, we will provide a quotation for repair or replacement, including return shipping fees.

Third-Party Repairs

This warranty complies with the federal Magnuson–Moss Warranty Act. You may use third-party parts or services for repairs without voiding this warranty, unless the repair itself causes damage to the product. We encourage you to retain your proof of purchase and contact our customer service team with any questions.

Unauthorized Sellers

This warranty applies only to products purchased from Beurer or authorized retailers and distributors. Products purchased from unauthorized sellers — including third-party platforms or online marketplaces (such as eBay or unverified Amazon sellers) — are not covered by this warranty. This policy is in place to protect customers from counterfeit, altered, or used products that may not meet Beurer's quality standards. If you're unsure whether your seller is authorized, please contact our support team.

Exclusions

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessories; alterations; improper installation; misapplication; lack of reasonable care; unauthorized repairs or modifications; improper electrical/power supply; worn batteries; normal wear and tear; power loss; dropped product; failure to follow instructions; lack of recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions.

This warranty is void if the product is used in a commercial or business environment. Beurer's maximum liability under this warranty is limited to the actual purchase price paid, as confirmed by proof of purchase. The warranty is effective only in the country where the product was purchased and used.

Legal Notice

This warranty gives you specific legal rights, and you may have other rights which vary from state to state.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE LENGTH OF THIS WRITTEN WARRANTY.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING FROM USE OR INABILITY TO USE THE PRODUCT.

Beurer does not authorize any third party — including retailers or resellers — to modify the terms of this warranty.

This warranty does not apply to opened, used, repackaged, or resold products, including those sold through online auctions or unauthorized channels.

For Canada only:
Manufacturer
Beurer GmbH
Söflinger Str. 218
89077 Ulm, Germany

Distributed by:
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020, USA
www.shop-beurer.com

Questions or comments? Call our US-based customer service toll free at
1-800-536-0366 or contact info@beurer.com.

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