beurer

EM 29

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**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING
THIS PRODUCT**

Keep this manual in a safe location for future reference

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Contents

1. IMPORTANT SAFETY INSTRUCTIONS.....	4	7. Troubleshooting Guide	13
2. Introduction	9	8. Technical Specifications.....	14
3. Package Contents.....	9	9. FCC Compliance Information.....	15
4. Parts and Controls.....	10	10. Warranty	15
6. Care, Maintenance and Disposal	12		

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact customer Beurer service. See warranty for service contact.

Dear Valued Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for application in the areas of heating, weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Register your product today to receive further benefits:

- added protection for your purchase
- health and wellness tips

- new product information
- continual outstanding customer service and support.

Visit www.registerbeurer.com or contact customer service at **1-800-536-0366**. Please read these instructions for use carefully and keep them for later use. Make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer team


1. IMPORTANT SAFETY INSTRUCTIONS



Signs and symbols

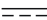Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

READ THIS ENTIRE MANUAL, THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

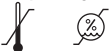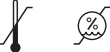	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
WARNING	Indicates a potentially hazardous situation, which, if not avoided, could result in death or serious injury.
CAUTION	Indicates a potentially hazardous situation, which, if not avoided, may result in minor/moderate injury.

NOTICE	Addresses practices not related to personal injury, such as product and/or property damage.
	IMPORTANT: The device can emit effective output values above 10 mA, averaged over every five-second interval
	Note Note on important information.
	Do not use the device with a pacemaker.
	Typ BF applied part (cuff only).
	Direct current
	Follow instructions for use.
	Keep away from rain
	Serial number

<p>Operating</p> 	Operating conditions
<p>Storage</p> 	Storage conditions
	Do not wash.
	Do not bleach.
	Do not tumble dry.
	Do not iron.
	Do not dry clean.

 **WARNING**

- Only use the knee and elbow TENS:
 - On humans
 - For external use
 - For the intended purpose and as specified in these instructions for use. Any form of improper use can be dangerous.
- In the event of an acute emergency, the provision of first aid has top priority.
- This device is not intended for commercial or clinical use; it is designed exclusively for self-treatment in a private home.
- Before use, ensure that there is no visible damage to the device or accessories. If you have any doubts, do not use the device and contact Customer Service.
- If you have health concerns of any kind, consult your doctor.
- The knee and elbow TENS is for external use on human knees or elbows only. Using the device on other parts of the body can lead to serious health problems.
- Do not use this product on the head, neck, or throat area, or over the ribcage. Use in any of these areas could possibly increase the risk of cardiac arrest, seizures, or suffocation.
- Slight reddening of the skin after use is normal and will subside after a short period. Do not use the device again until the reddening has disappeared.

- If skin becomes irritated over longer treatment times, select a shorter application time instead.
- If more serious skin irritation occurs, stop the treatment and seek medical assistance.
- This device is not intended for use by children or people with restricted physical, sensory (e.g. reduced sensitivity to pain) or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person on how to use the device.
- Keep packaging material away from children and pets (risk of suffocation).
- Store the device and accessories out of the reach of children. There is a risk of strangulation from the connection cable.
- Do not use any additional parts that are not recommended by the manufacturer.
- Do not move the cuff or put it on if the control unit is switched on.
- Only connect the connection cable for the electrode plug to the intended control unit and the universal cuff that is included.
- Do not pull on the connection cables during treatment.
- Do not bend or pull on the end of the connection cable.
- During the initial few minutes, use the device while sitting or lying down to minimize the risk of injuries as a result of isolated cases of vagal responses (feeling of faintness). If you feel faint, switch off the device immediately, lie down and support the legs in an elevated position (approx. 5-10 min).
- The treatment should be comfortable. If the device does not work properly, or you feel unwell or experience pain, stop using it immediately.
- Only remove the cuff once the device has been switched off.
- Do not wear any electronic devices, such as watches, while using the device.
- Do not use the device near (~ 3 ft) shortwave or microwave devices (such as mobile phones) as these may cause fluctuations in the output values of the device.
- Never immerse the device in water or other liquids.
- Keep the device away from sources of heat.
- Do not use in the vicinity of highly flammable substances, gases, or explosives.
- The device is suitable for self-treatment.
- For hygiene reasons, the cuff may only be used on one person.
- Ensure that no metallic objects contact the electrodes during stimulation. Failure to do so could result in spot burns.
- Do not use the device in areas with high humidity such as in the bathroom or when bathing or showering.
- The electrodes must not be attached to open areas of skin.
- Pre-treating the skin with greasy creams or lotions is not recommended.

- Do not modify the device or cuff as this is potentially hazardous.
- Max. recommended output value for electrodes is 5 mA/cm².
- Effective current densities over 2 mA/cm² require increased attentiveness from the user.

This device is for external use on human knees and elbows only. Do not use this device in the following situations:

- Do not use the device if you have a pacemaker or other implants, such as an insulin pump or metal implants.
- If you have a high temperature (e.g. >102° F /39° C).
- If you have a known or acute cardiac arrhythmia, or disorders of the heart's impulse and conduction system.
- If you suffer from a seizure disorder (e.g. epilepsy).
- If you are pregnant.
- If you have cancer.
- If you suffer from metabolic arthritis.
- After an operation, if strong muscle contractions could affect the healing process.
- On surgical scars that are still healing.
- On acutely or chronically affected (injured or irritated) skin, e.g. inflamed skin, whether painful or not, or reddened skin.
- On rashes (e.g. allergies), burns, bruises, swellings, as well as open and healing wounds.
- If you are connected to a high-frequency surgical device. This may lead to burns under the electrical muscle stimulation areas.

- Under the influence of pain-relieving medication, alcohol, or sleeping tablets.
- In the case of acute or chronic diseases of the gastrointestinal tract.
- While undertaking any activity where an unexpected reaction (e.g. strong muscle contractions even at low intensity) could be dangerous, e.g. while driving or operating/driving machinery.
- On a sleeping person.
- Do not use this device while using other devices that transmit electrical impulses into your body.

Consult your doctor before use if you have:

- A serious illness, in particular if you suspect or have been diagnosed with a blood coagulation disorder, propensity to thromboembolic conditions, or recurrent malignant growths.
- Diabetes or other health conditions.
- Unexplained chronic pain in any part of the body.
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- Medical treatment.
- Complaints linked to stimulation treatment.
- Skin diseases or open wounds.
- Persistently irritated skin under the electrodes.

NOTICE:

- Protect the device from dust, dirt, and humidity.
- If the device has been dropped or exposed to high levels of humidity or has suffered any other damage, discontinue use. The device must not be exposed to high temperatures or direct sunlight.
- Under no circumstances should you open or repair the device yourself because faultless functionality can no longer be guaranteed thereafter. Failure to comply will result in voiding of the warranty.
- For repairs, contact Customer Service.

Battery Handling Safety Precautions

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.
- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions.)
- Do not dispose of batteries in fire, batteries may explode or leak.

- Clean the battery contacts and also those of the device prior to battery installation.
- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.

Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

2. Introduction

How does electrical muscle stimulation work?

The knee and elbow TENS use a method called Transcutaneous Electrical Nerve Stimulation (TENS), which electrically stimulates nerves through the skin. It is an effective non-pharmacological method of treating different types of pain from a variety of causes. It has no side effects if administered correctly. The method has been clinically tested and approved and can be used for simple self-treatment. The pain-relieving or pain-suppressing effect is achieved by inhibiting the transference of pain to nerve fibers (caused mainly by high-frequency impulses) and by increasing the secretion of endorphins in the body. Their effect on the central nervous system reduces the sensation of pain. This method is clinically tested and approved. Any symptoms that can be relieved using TENS should be checked by your doctor. Your doctor will also give you instructions on how to carry out a TENS self-treatment regime.


3. Package Contents

Check that the exterior of the delivery cardboard packaging is intact and make sure that all components are present. Before use, ensure that the device and accessories show no visible damage and all packaging material has been removed. If you have any doubts, do not use the device

and contact your retailer or the specified Customer Service address.

- 1 x Knee and elbow universal cuff
- 1 x Control unit
- 1 x Connection cable
- 3 x 1.5 V AAA Batteries (LR03)
- 1 x Instruction Manual

4. Parts and Controls

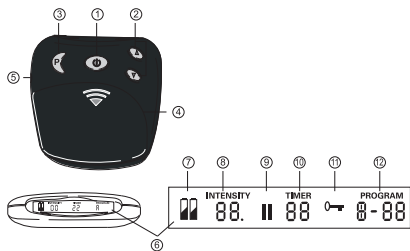No.	Designation
1	On/Off/Pause button
2	Intensity setting ▲ Increase ▼ Lower
3	Program/Lock button
4	Battery Compartment
5	Connector Socket
6	LCD display
7	Low Battery Indicator
8	Intensity Level (0-20)

9	Pause Symbol, flashes when activated
10	TIMER: Remaining time in the active program in minutes
11	Button Lock Indicator
12	Active Program Number

5. To Use



Insert/Change Batteries:

1. Open the battery compartment by pushing the cover in the area marked with an arrow and sliding downwards.
2. Insert three AAA alkaline batteries, observing the correct polarity indicated inside the battery compartment.
3. Close the battery compartment lid until it clicks into place.
4. When the Low Battery Icon appears in the LCD Display, replace the batteries using the steps described above.

Connecting Components:

1. Attach the electrode cables to the cuff. Do not pull, twist, or make sharp bends in the cables.
2. Before positioning the Cuff on either your knee or elbow, moisten the electrodes and your knee/elbow with water. Position the Cuff making sure the electrodes surround the knee/elbow. Note that the round opening of the Cuff should be over the kneecap/elbow.
3. Secure the Cuff in the desired position using the hook-and-loop fasteners. Ensure the Cuff is not too tight but that the electrodes have sufficient skin contact.

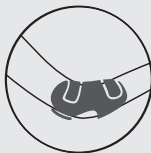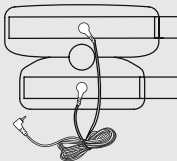4. Attach the cable plug to the connector socket on the Control Unit.

5. Press and hold the ON/OFF Button until the unit beeps and the LCD Display switches on. When the device powers on for the first time, program "A" is activated.
6. To increase or decrease the intensity at any time during treatment, press and hold the Increase/Decrease Intensity Buttons, as applicable, until the desired intensity level is reached. This device has 20 intensity levels to choose from. Depending on the intensity level, you may experience a tingling sensation in the treatment area, up to and including muscle contractions.
7. To prevent accidental changes in intensity, press and hold the Lock Button (P) until you hear a beep and the Lock Icon (🔒) appears in the LCD Display. Press and hold the Lock Button again to unlock.

Selecting a program

Press the Program (P) Button to change programs

Program	Frequency	Time
A	4 Hz – 110 Hz (3 phases)	30 min
B	4 Hz	25 min
C	2 Hz (burst)	25 min
D	100 Hz	25 min

NOTES:

- When using Program A, the effect will intensify when switching to Phase 2 (after approx. 10 minutes); this is normal. If this intensity is too high, lower it by pressing the Intensity Decrease Button (▼).
- If the program is changed during stimulation (for example from A to B), the output intensity will increase gradually in the new program until it reaches the previously set level. To cancel, press and hold the Intensity Decrease Button (▼) for two seconds or shut off the device (press and hold the ON/OFF Button).
- If you experience unpleasant twinges or itchiness, shut the device off and check for improper use/damage, such as electrode placement/damage, electrode covers missing/damaged, and proper electrode/skin dampness.

- The device will not operate (and intensity levels cannot be adjusted) if the electrodes are not making proper contact with the skin.
- The device will save the last set program. After battery replacement, the device will default to the first program.

Pausing a Program:

To pause a program while it is running, press the Pause Button; you will hear a beep and the Pause icon will flash in the LCD Display. To continue, press the Pause Button again.

6. Care, Maintenance and Disposal

CAUTION:

Before cleaning, remove the cable from the control unit and cuff and remove the batteries.

Cleaning

Clean the control unit after each use with a soft, slightly damp cloth. For more thorough cleaning, moisten the cloth with a mild soapy solution. Do not use chemical or abrasive cleaning agents. Keep water away from the inside of the Control Unit.

Clean the Cuff carefully in lukewarm, soapy water; do not use hot water. Rinse well with clean water until no soap residue remains. Carefully pat the Cuff dry with a towel and let dry completely before next use.

Storage

Shut the Control Unit off and disconnect the cable from the Control Unit and Cuff. If not using the device for a long period, remove the batteries. Place the Control Unit, Cuff, and Cable in the original packaging and store in a cool, dry place away from children.

Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

7. Troubleshooting Guide

Problem	Solution
The device does not switch on when the ON/OFF button is pressed.	Ensure batteries have been inserted correctly and are in contact with the terminals.
	Replace batteries, if applicable.
	Contact Customer Service.

Problem	Solution
Electrodes do not adhere to the body.	Clean the adhesive surface of the electrodes using a damp, lint-free cloth. Replace the electrodes if they still do not adhere securely.
	Clean the skin prior to any application; do not use skincare lotions or oils prior to treatment. Shaving may increase the life of electrodes.
There is no noticeable stimulation.	Press the ON/OFF Button to interrupt the program. Ensure the cables are correctly connected to the electrodes. Ensure the Cuff is in firm contact with the treatment area.
	Ensure the electrodes are sufficiently moist.
	Press the ON/OFF button to restart the program.
	The batteries are almost empty. Replace the batteries.
The low battery symbol is shown.	Replace batteries.

Problem	Solution
You have an unpleasant/irregular sensation in the treatment area.	The intensity on the device may be set too high; lower the intensity.
	Ensure electrodes are sufficiently moist.
	Ensure cable is not worn/faulty; replace if necessary.
	Ensure electrodes are not worn/faulty; replace Cuff if necessary.
Skin in the treatment area turns red or you experience a stabbing sensation.	Immediately stop treatment and wait until your skin has returned to its normal condition. If the redness is under the electrode and disappears quickly, there is no risk, as this is caused by locally stimulated, increased blood flow.
	Consult your doctor, however, before you continue treatment if the skin irritation persists and is accompanied by an itchy sensation or inflammation.
	Ensure cuff is clean and positioned correctly. Ensure electrodes are not scratched or damaged and are sufficiently moist.

Problem	Solution
Stimulation stops during treatment.	Cuff has repositioned and is not contacting the skin properly; reposition cuff. Ensure cables are still attached securely and that batteries are not depleted.

8. Technical Specifications

Type	EM 29
Cuff circumference	9.8 - 27.5 inches (25-70 cm)
Output waveform	Symmetric biphasic rectangular pulse
Pulse length	60 – 220 μ s per phase
Pulse frequency	2 - 110 Hz
Output voltage max.	50 Vpp (500 ohm load)
Output current max.	100 mApp (500 ohm load)
Voltage supply	3 AAA batteries
Intensity Levels	Adjustable from 0 to 20
Operating conditions	32 °F - 104 °F (0 °C - 40 °C) at a relative humidity of 20 - 65%
Storage conditions	32 °F - 131 °F (0 °C - 55 °C) at a relative humidity of 10 - 90%

Transport conditions:	14 °F - 122 °F (-10 °C - 50 °C)
Weight	Device: 3.3 oz. (95 g) incl. batteries; Device, Cuff, and Cable: 7.4 oz. (210 g)

9. FCC Compliance Information

Knee and Elbow TENS device EM 29

Responsible Party – U.S. Contact Information

Beurer North America LP

1 Oakwood Boulevard, Suite 255

Hollywood, FL 33020

United States

1-800-536-0366

info@beurer.com

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

(1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

10. Warranty

Your Beurer Knee and Elbow TENS device, model EM 29, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty

extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Knee and Elbow TENS device, model EM 29, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal

wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS

FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.

Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state. For more information regarding our product line in the USA, please visit: www.beurer.com

Distributed by:
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020
USA
www.shop-beurer.com

Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.

German Design and Engineering
Assembled in China

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