# beurer EM 50 Menstrual Relief

- Questions or Comments? Call our US-based customer service toll free at 1-800-536-0366.
- ¿Preguntas o comentarios? Llame a nuestro servicio al cliente en Estados Unidos gratis al teléfono 1-800-536-0366.
- Questions ou commentaires? Appelez gratuitement notre service clientèle situé aux États-Unis au 1-800-536-0366.

For Canada only / Solo para Canadá / Pour le Canada seulement:  
Manufacturer / Fabricante / Fabricant : Beurer GmbH, Söflinger Str. 218, 89077 Ulm, Germany  
Distributed by / Distribuido por / Distribué par :  
Beurer North America LP, 1 Oakwood Boulevard, Suite 255, Hollywood, FL 33020, USA www.beurer.com

## ENGLISH

### READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT

Keep this manual in a safe location for future reference

### SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact customer service. See warranty for service contact.

Dear Valued Customer,  
Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for application in the areas of heating, weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Register your product today to receive further benefits:  
• added protection for your purchase  
• health and wellness tips  
• new product information  
• continual outstanding customer service and support.

Visit [www.registrierbeurer.com](http://www.registrierbeurer.com) or contact customer service at 1-800-536-0366. Please read these instructions for use carefully and keep them for later use. Make them accessible to other users and observe the information they contain.

With kind regards,  
Your Beurer team

## 1. Important safety notes

### Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.  
**READ THIS ENTIRE MANUAL, THE AFFECTED AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.**

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
	Indicates a potentially hazardous situation, which, if not avoided, could result in death or serious injury.
	Indicates a potentially hazardous situation, which, if not avoided, may result in minor/moderate injury.
	Addresses practices not related to personal injury, such as product and/or property damage.
	<b>IMPORTANT</b> The device can emit effective output values above 10 mA, averaged over every 5-second interval.
	<b>Isolation of applied parts Type BF</b> Galvanically isolated applied part (F stands for "floating"); meets the requirements for leakage currents for type B.
	Do not use the device with a pacemaker.
	Do not use near the heart.
	Device protected against foreign objects > 12.5 mm and against water dripping at an angle.
	Expiry date
	Observe the instructions
	Permissible storage temperature and humidity
	Permissible operating temperature and humidity
	Serial Number

### WARNING – To reduce the risk of fire, electric shock, or serious personal injury:

- Only use this device for the purpose for which it was developed and in the manner specified. Any improper use can be hazardous.
- Remove all jewelry or body piercings before using the device.
- Do not use this device at the same time as other devices that emit electrical pulses, such as TENS devices.
- The device is only for private use and is not intended for medical or commercial purposes.
- The manufacturer assumes no liability for damages caused through improper or wrong use.
- If you are in any doubt, do not use and contact customer service.
- Only use the device with the accessory parts supplied.
- For hygienic reasons, use the device only on one person.
- In rare instances, skin irritation in the region of the electrodes may occur.
- The device is only designed for use by healthy adults. Keep the device away from children.
- Keep the device away from heat sources.
- Never use the device near microwave ovens or microwave devices (e.g. cell phones) as they can affect operation.
- Never use the device while you bathe or shower. Do not store or keep the device where it can fall into a bathtub or sink.
- Check the device regularly for wear and damage. If you find such signs, discontinue use and contact customer service.
- Switch off the device immediately if it malfunctions.
- If the device does not work properly or if you feel unwell or experience pain, stop using it immediately.
- Switch off the device before attaching or removing it.
- Do not modify the device in any way (e.g. by cutting it), as this will produce a higher current density and may be dangerous.
- During the initial few minutes, use the device while sitting or lying down, to minimize the risk of accidental injuries arising from rapid responses (feeling of faintness) that have been infrequently reported as a side effect. If you feel faint, switch off the device immediately, lie down and support the legs in an elevated position (approx. 5–10 minutes).
- Do not use this device if you have any of the following:
  - Implanted electrical devices (e.g. pacemakers)

- Metal implants near the application site
- Cardiac arrhythmia
- Acute illness
- Epilepsy
- Malignant diseases
- Skin injuries or cuts
- Pregnancy
- Insulin Pump
- An increased tendency to bleed, e.g. after an acute injury or a fracture. The stimulation current can trigger or exacerbate bleeding.
- Recent operation when muscle contractions could impair the healing process
- A high frequency surgical device connected
- Low or high blood pressure
- High fever
- Psychosis
- Swollen or inflamed regions

- To avoid damage to health, we strongly advise NOT using this device in the following situations:
  - Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant growths.
  - Any skin conditions.
  - Unexplained chronic pain in any part of the body.
  - Diabetes.
  - Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
  - Receiving medical treatment.
  - In the event of complaints linked to stimulation treatment.
  - Suffering from persistently irritated skin due to long-term stimulation at the same site.
- Do not use this device:
  - On the head, as it could cause seizures.
  - On the neck/carotid artery, as it could cause cardiac arrest.
  - On the pharynx or larynx, as it could cause muscle spasms leading to suffocation.
  - Near the upper chest, since it could increase the risk of ventricular fibrillation leading to cardiac arrest.
  - In the genital area.
- Do not use when asleep, when driving a vehicle or when operating machinery.
- Do not use the device after consuming alcohol.
- Do not use the device in cases of acute or chronic diseases of the gastrointestinal tract.
- Do not use while undertaking any activity where an unexpected reaction (e.g. strong muscle contractions even at low intensity) could be dangerous.
- The actual temperature of the device may vary depending on the condition of your skin, your age, the location of the pain etc.
- If the heat function feels too hot, stop treatment immediately. You can continue the TENS treatment without the heat function.
- Wait at least one hour after eating before starting treatment with the device.
- Do not use the device on wet skin as this impairs the adhesion of the gel pads.
- Pre-treating the skin with greasy creams or lotions is not recommended.
- Do not use adhesive tapes, bandages or any medium other than the gel pads provided to adhere the device to your skin, as the uneven adhesion of the gel pads could lead to skin lesions.
- If in doubt, consult your doctor.

• To avoid damage to health, we strongly advise NOT using this device in the following situations:

- Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant growths.
- Any skin conditions.
- Unexplained chronic pain in any part of the body.
- Diabetes.
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- Receiving medical treatment.
- In the event of complaints linked to stimulation treatment.
- Suffering from persistently irritated skin due to long-term stimulation at the same site.

• To avoid damage to health, we strongly advise NOT using this device in the following situations:

- Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant growths.
- Any skin conditions.
- Unexplained chronic pain in any part of the body.
- Diabetes.
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- Receiving medical treatment.
- In the event of complaints linked to stimulation treatment.
- Suffering from persistently irritated skin due to long-term stimulation at the same site.

• To avoid damage to health, we strongly advise NOT using this device in the following situations:

- Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant growths.
- Any skin conditions.
- Unexplained chronic pain in any part of the body.
- Diabetes.
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- Receiving medical treatment.
- In the event of complaints linked to stimulation treatment.
- Suffering from persistently irritated skin due to long-term stimulation at the same site.

## 2. Introduction

### What is the EM 50 Menstrual Relief?

Many women experience abdominal cramps and severe period pain before and during their period. Many women also suffer from illnesses that cause lower abdominal pain, which is often so strong that women seek immediate pain relief. The innovative Menstrual Relief is a self-adhesive TENS device with heat function to provide pain relief for menstrual discomfort and pain. The TENS technology used in the Menstrual Relief enables targeted pain relief treatment with 15 intensity levels and features a soothing heat function.

Made with medical-quality silicone, the Menstrual Relief is slim, flexible, and adapts perfectly to treatment areas. The high-tech circuit provides energy-efficient 20-minute long treatments (up to six treatments per battery charge). The Menstrual Relief is a safe, easy, and innovative way to treat lower abdominal pain.

### What is TENS and what can it do?

Transcutaneous Electrical Nerve Stimulation (TENS) involves electrical stimulation of nerves through the skin. TENS is an effective non-pharmacological method for treating different types of pain from a variety of causes and has no side effects when administered correctly. This method has been clinically tested and approved and can be used for simple self-treatment. The pain-relieving or pain-suppressing effect is achieved by inhibiting the transfer of pain to nerve fibers and increasing the secretion of endorphins in the body, as their effect on the central nervous system reduces the sensation of pain. The method is scientifically substantiated and approved as a form of medical treatment.

Any symptoms treatable using TENS should be checked by your physician, who will give you instructions on how to initiate a TENS self-treatment program. TENS has been clinically tested and approved for use in cases of menstrual discomfort. TENS devices pass electrical currents over the skin via gel pads, which are subject to natural wear and tear. Replace gel pads when they no longer adhere to your skin properly. Failure to replace the pads may lead to skin irritation from increased current in particular areas.


## 3. Package Contents

Check that the exterior of the delivery cardboard packaging is intact and make sure that all components are present. Before use, ensure that the device and accessories show no visible damage and all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 x Menstrual Relief EM 50
- 2 x self-adhesive gel pads
- 1 USB-C cable
- 1 Instruction manual

## 4. Parts and Controls

- Intensity Up Button
- Heat Level/Battery Level Indicator LED
- TENS LED
- Power Button
- Intensity Down Button
- Heating Pad
- Electrodes
- USB-C Port

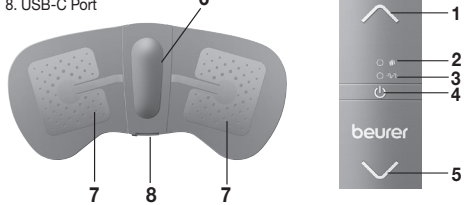## 5. To Use

### Charging the Battery:

Before first use, and whenever the Low Battery Indicator LED flashes orange, the internal battery pack must be charged for at least two hours.

1. Open the USB-C Port cover and attach the provided USB-C cable. Connect the other end of the cable to a USB power source, such as a computer or suitable AC Adapter (V/DVC, 2A output, not included).
2. The Battery Level Indicator LED will flash green while the device is charging and will illuminate steadily when charging is complete.

**NOTE:** The device cannot be used while it is charging.

### Attaching the Gel Pads:

**WARNING:** To avoid serious personal injury, use the device only around the abdomen. Do not use on head, neck, or chest area.

1. Before each treatment session, ensure the skin underneath the device is clean and free of creams or oils.
2. Lay the device down with the electrodes facing up. Take a gel pad, remove one of the protective films from the adhesive, and attach the exposed adhesive side of the gel pad to one of the electrodes. Remove the other protective film.
3. Repeat step 2 for the other gel pad/electrode.

### Treatment

This device has 15 intensity levels and an on/off heat function. Begin treatment using the lowest intensity level and increase as your comfort level allows. The device also contains an auto shutoff feature that will shut it off after 20 minutes of use.

1. Attach the device securely to your lower abdomen. Note that the device can detect whether it is in good contact with your body and will beep and shut off if you attempt to use it without sufficient skin contact. If the device is attached securely and still will not operate properly, shut it off and clean the gel pads before trying again.
2. Press and hold the Power Button to activate the device. The Heat Indicator LED will flash green and the device will begin to heat up.
3. Press the Power Button again to activate the TENS function; the TENS Indicator LED will flash. To begin treatment, press the Intensity Up Button to activate TENS at level one.
4. To cancel the Heat Function, press the Power Button again; the Heat Indicator LED will shut off.
5. To increase intensity, press the Intensity Up Button repeatedly until the desired level is achieved. To decrease the intensity, press the Intensity Down button.
6. When treatment is finished, press and hold the Power Button to shut the device off (if the auto shutoff was not activated).

After treatment, carefully remove the device from your abdomen and reattach the protective films to the exposed adhesive surfaces of the gel pads.

**NOTE:** Wait at least 30 minutes before starting another treatment session.

**CAUTION:**

- If you experience unpleasant electrical current spikes, shut the device off and reposition it.
- If you find that your menstrual flow becomes much heavier after treatments, discontinue use.

## Program

Cycle	1
Pulse width	100 µs
Pulse frequency	100 Hz

### Pulse frequency

The frequency indicates the number of individual impulses per second and is displayed in Hz (Hertz). It can be calculated by determining the cyclic value for the time period.

### Pulse width

The pulse width indicates the duration of an individual impulse in microseconds. The pulse width therefore determines, among other things, the penetration of the electricity, which usually larger muscle mass requires a larger pulse width.

## 6. Care, Maintenance and Disposal

- Clean the device with a slightly moistened cloth. In case of more extreme soiling you can also moisten the cloth with mild soapy water.
- Never drop the device.
- Prevent water from entering the device. If water reaches any internal parts, dry them completely before using again.
- Do not use any harsh chemicals or abrasive cleaners.
- Do not machine wash.

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• To avoid damage to health, we strongly advise NOT using this device in the following situations:

• Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant growths.

• Any skin conditions.

• Unexplained chronic pain in any part of the body.

• Diabetes.

• Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).

• Receiving medical treatment.

• In the event of complaints linked to stimulation treatment.

• Suffering from persistently irritated skin due to long-term stimulation at the same site.

• To avoid damage to health, we strongly advise NOT using this device in the following situations:

• Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant growths.

• Any skin conditions.

• Unexplained chronic pain in any part of the body.

• Diabetes.

• Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).

• Receiving medical treatment.

• In the event of complaints linked to stimulation treatment.

• Suffering from persistently irritated skin due to long-term stimulation at the same site.

• To avoid damage to health, we strongly advise NOT using this device in the following situations:

• Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant growths.

• Any skin conditions.

• Unexplained chronic pain in any part of the body.

• Diabetes.

• Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).

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Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

## 7. Troubleshooting Guide

Problem	Possible cause	Solution
The device does not switch on when the ON/OFF button is pressed.	The battery may be depleted.	Charge the battery. If the device still does not work, contact Customer Service.
Gel pads do not adhere to the skin.	The gel pads may be dirty or worn. Skin in the treatment area may be rough or not clean enough.	Clean the surface of the gel pads with a damp, lint-free cloth. If adhesiveness does not improve, replace gel pads. Clean the skin before treatment. Do not use lotions or oils. Shave area if necessary.
Cannot feel stimulation in treatment area.	The gel pads may be dirty or worn. The battery may be depleted.	Clean pads as described above. Charge battery.
Unpleasant sensation felt underneath gel pads.	Pads are worn and cannot evenly distribute current.	Replace the gel pads.
Skin in the treatment area turns red.	Redness can be caused by increased blood flow in the treatment area caused by the device; this is normal.	Redness should go away a few minutes after treatment. If redness persists, or if you experience any itching or inflammation, contact your doctor.
The Menstrual Relief becomes too hot.	Treatment duration too long.	Press the ON/OFF button to shut off the heat function and continue with TENS treatment only.

## 8. Technical Specifications

Model	EM 50 Menstrual Relief
Type	FT610-B
Output waveform	Biphasic rectangular pulse
Pulse length	100 µs
Pulse frequency	100 Hz
Output voltage	Max. 110 V <sub>pp</sub> +/- 10 <span> </span> % (at 1000 Ω)
Heat output	109 <span> </span> °F (43 <span> </span> °C)
Output current	Max. 110 mA <sub>pp</sub> +/- 10 <span> </span> % (at 1000 Ω)
Voltage supply	320 mAh lithium polymer battery
Treatment time	20 minutes to auto shutoff
Intensity Levels	15
Operating conditions	41° - 104° F (5° - 40° C) at 15 - 90% relative humidity
Storage conditions	32° - 104 °F (0° - 40° C) at 0 - 90 <span> </span> % relative humidity
Dimensions	3.7 x 7.6 x 0.6 inches (95 x 193 x 15 mm)
Weight	1.6 oz (47 g)

### 9. FCC Compliance Information

Menstrual Relief EM 50	<i>Sous réserve d'erreur et de modification</i>
Responsible Party – U.S. Contact Information	
Beurer North America LP 1 Oakwood Boulevard, Suite 255 Hollywood, FL 33020 United States 1-800-536-0366 info@beurer.com	

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures: Reorient or relocate the receiving antenna. Increase the separation between the equipment and receiver. Connect the equipment into an outlet on a circuit different from that to which the receiver is connected. Consult the dealer or an experienced radio/TV technician for help. Warranty Your Beurer Menstrual Relief, model EM 50, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Menstrual Relief, model EM 50, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your cost in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope

of the limited lifetime warranty, you will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY, ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

**IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLICIT OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.** Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you. Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resaled products, including but not limited to sale of such products on internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state. For more information regarding our product line in the USA, please visit: www.beurer.com

Distributed by:
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020, USA
www.beurer.com

Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.

Assembled in China

#### Annex 1 – Electromagnetic Compatibility

**A1.1 Important information regarding Electro Magnetic Compatibility (EMC)**

With the increased number of electronic devices such as computers and mobile (cellular) telephones, this device in use may be susceptible to electromagnetic interference from other devices, even if it complies with CISPR emission requirements. Electromagnetic interference may result in incorrect operation of this device and create a potentially unsafe situation. Besides, this device should also not interfere with other devices. In order to regulate the requirements for EMC (Electro Magnetic Compatibility) with the aim to prevent unsafe product situations, the IEC 60601-1-2 standard has been implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions for medical devices. This device conforms to this IEC 60601-1-2 standard for both immunity and emissions. Nevertheless, special precautions need to be observed: The use of the device adjacent to or stacked with other device should be avoided because dot could result in improper operation. In case such use is necessary, the device and other devices should be observed to verify that they dare operating normally?
• The use of any parts other than those specified or provided could result inincreased electromagnetic emissions or decreased electromagnetic immunity of this deviceand result in improper operation.

• Portable RF communications device (including peripherals such as antenna cables and external antenna) should be used no closer than 12 inches (30 cm) to any part of this device. Otherwise, degradation of the performance of the device could result.

• Refer to further guidance below regarding the EMC environment in which the device shouldbe used.

**A1.2 Guidance and manufacturer's declaration – Electromagnetic emission**

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Test Standard	Test Item	Test Level	Electromagnetic environment – Guidance
EM 55011 / CISPR 11	Conducted Emissions of Powerline	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
EM 55011 / CISPR 11	Radiated Emissions	Class B	This device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
EN/IEC 61000-3-2	Harmonic Current Emission	Not applicable (Note 1)	–
EN/IEC 61000-3-3	Voltage Fluctuations and Flicker	Not applicable (Note 1)	–
Note 1:	The power supply of this device is from battery (DC voltage), so Harmonic current emission, Voltage fluctuations & flicker, EFT, Surge, CS and DIP tests are not applicable for this device.		
Note 2:	Standards applicable to other modes or EM ENVIRONMENTS of transportation for which use is intended shall apply.		

**A1.3 Guidance and manufacturer's declaration – Electromagnetic immunity**

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Test Standard	Test Item	Test Level	Electromagnetic environment – Guidance
IEC 61000-4-2	Electrostatic discharge	Contact: ± 8 kV, Air-2 kV, ±4 kV, ±6 kV, and ±15 kV	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%. Portable and mobile RF communications equipment should be used no closer than 30 cm (12 inch) to any part of this device.
IEC 61000-4-3	Radiated RF EM fields	10 V/m; 80 MHz to 2.7 GHz; 80% AM at 1 kHz	Portable and mobile RF communications equipment should be used no closer than 30 cm (12 inch) to any part of this device.
IEC 61000-4-3	Proximity fields from RF wireless communications equipment	Not applicable (Note 1)	–
IEC 61000-4-4	Electrical test transients / bursts	Not applicable (Note 1)	–
IEC 61000-4-5	Surges line-to-line	Not applicable (Note 1)	–