beurer

FT 90

**READ THIS MANUAL COMPLETELY AND
CAREFULLY BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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ENGLISH **Non-contact Forehead Thermometer**
Instruction manual 2

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Questions or Comments? Call our US-based customer service toll free at 1-800-536-0366.

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ENGLISH

Content

1. Important safety notes	3	8. Measuring Object Temperature/Room Temperature.....	16
2. Package Contents	7	9. Troubleshooting Guide	17
3. Parts and Controls.....	8	10. Changing batteries	17
4. Introduction	9	11. Care, Maintenance and Disposal	18
5. Initial Use.....	11	12. Technical Specifications.....	19
6. Switching On and Setting The Thermometer	11	13. FCC Compliance Information	20
7. Forehead Measurement.....	14	14. Warranty	21

Dear Valued Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for application in the areas of heating, weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Register your product today to receive further benefits:

- added protection for your purchase
- health and wellness tips
- new product information
- continual outstanding customer service and support.

Visit www.registerbeurer.com or contact customer service at 1-800-536-0366.

Please read these instructions for use carefully and keep them for later use.
Make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer Team

SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Indication for use

The non-contact forehead thermometer, model FT 90, is an infrared thermometer intended for the periodic measurement of human body temperature for persons of all ages. Additionally it can measure the room and object temperature in seconds.

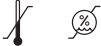
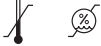
1. Important safety notes

Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

READ THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

 WARNING	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
 CAUTION	CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
NOTICE	NOTICE addresses practices not related to personal injury, such as product and/or property damage.
	Type BF applied part

	Follow instructions for use
Operating 	Permissible operating temperature and humidity
Storage 	Permissible storage temperature and humidity
SN	Serial number
	Protect from Moisture

WARNING:

- Prolonged high fever requires medical attention-especially in young children.
- The thermometer needs to be in the room in which the measurement is taken for at least 30 minutes before use.
- The thermometer is only designed for the measuring point on the human body stated in the instructions for use.
- The device is only intended for the purpose stated in these instructions for use.
- Children must not be allowed to use the device. Medical products are no toys.
- The device has been designed for practical use, but is not a substitute for a visit to the doctor.
- Should you have any questions about using the device, please contact either your retailer or Customer Services.
- Packaging materials are a deadly hazard for children and can cause suffocation. Remove all packaging materials immediately and keep them away from children at all times.

- Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the battery out of the reach of children. Should a child, or any other person swallow a battery and/or battery fluid, please call 911 immediately.
- Should battery fluid leak and come into contact with your eyes or skin, immediately rinse with plenty of clean water. Please call 911 immediately.

CAUTION:

- Check before each use that the measuring sensor is intact. If it is damaged, please contact your retailer or the service address.
- This unit is intended for home use only. It is not intended for professional use in hospitals or other medical facilities. This unit does not support the standards required for professional use.
- Do not disassemble this unit.

NOTICE:

- Portable and mobile HF communication systems may interfere with this device. For more details, please contact our customer service at the address indicated or see the enclosed information sheet.
- Do not subject the device to mechanical impacts.
- When not in use, store the thermometer in the provided storage case.
- Do not expose the device to liquids. The device is not waterproof. Avoid all direct contact with water or other liquids.
- Do not expose the device to direct sunlight.
- Clean the measuring sensor before each use.
- Do not touch the measuring sensor with fingers.
- Do not use any aggressive solvents, cleaning agents, detergents or any other strong chemicals to clean the device.
- Replace weak batteries before they discharge completely. When “” in the display occurs please change the batteries.

- Due to anatomical differences in people, the (highest) temperature measured at different positions on the forehead may vary. When in doubt, move the thermometer over the entire forehead area during measurement.
- Do not use mobile telephones, microwaves, and other devices which generate strong electrical or electromagnetic fields near the unit. This may result in incorrect operation of the unit.
- Do not press the buttons with excessive force or with any pointed objects.
- Never attempt to repair the instrument or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our Customer Service. Please see warranty for service contact. Before submitting any complaint, first check the batteries and replace them if necessary.
- Changes or modifications to the device will nullify the user warranty.

Battery Handling Safety Precautions

- Use only the size and type of battery specified.
- Be sure to follow the correct polarity when installing the battery. Reversed battery may cause damage to the device.
- If the battery in the device is depleted or the device will not be used for a long period of time, remove the battery to prevent damage or injury from possible battery leakage.
- Do not try to recharge a non-rechargeable battery; it can overheat and rupture (follow battery manufacturer's directions.)
- Do not dispose of battery in fire, battery may explode or leak.
- Clean the battery contacts and also those of the device contacts prior to battery installation.
- Remove discharged battery from the product and dispose/recycle in compliance with all applicable laws.
- Keep battery away from children and pets. Battery may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.

Notes on electromagnetic compatibility

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

2. Package Contents

- Non-contact forehead thermometer FT 90
- 2 x AAA (LR03) batteries
- Storage box
- Quick start guide
- Instruction manual

3. Parts and Controls

- 1 Display
- 2 MODE/MEM button
- 3 On/Off button ①
- 4 LIGHT/SET button
- 5 Start-measurement button
- 6 Measuring sensor
- 7 Battery compartment lock
- 8 Battery compartment lid

Buttons

Start

MODE/MEM (short)

MODE/MEM (5 sec.)

LIGHT/SET (short)

LIGHT/SET (long)

Functions

Turns the device on and off.


Starts the temperature measurement.

Sets the measurement mode.

Displays stored measurements.


Manual switch on of the illuminated display.

Sets the basic functions.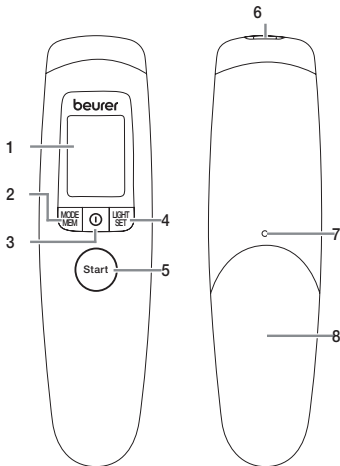

3.1 Display Description

- 1 Object temperature mode
- 2 Room temperature mode
- 3 Forehead temperature mode
- 4 Acoustic signal symbol
- 5 Year
- 6 Memory function
- 7 Date
- 8 Battery level
- 9 Temperature/memory space number display
- 10 Fahrenheit/Celsius measurement unit
- 11 Measurement result: "fever"
- 12 Measurement result: "no fever"
- 13 Illuminated display symbol
- 14 Year/date/time display

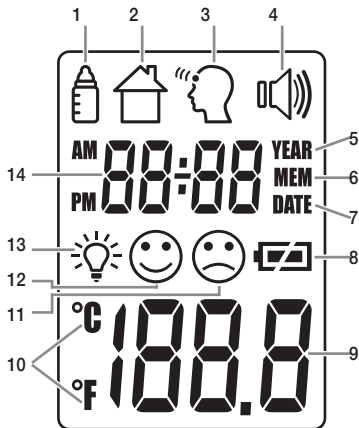4. Introduction

The temperature measurement varies depending on the part of the body where the measurement is taken. In a healthy person, the variance can be between 0.4°F and 1.8°F (0.2°C and 1°C) in different parts of the body.

The result of FT 90 is shown as oral temperature.

Normal temperature range with various thermometers

	Measurements	Thermometer used
Forehead temperature	96.4°F to 99.7°F (35.8°C to 37.6°C)	Forehead thermometer
Ear temperature	96.8°F to 100.0°F (36.0°C to 37.8°C)	Ear thermometer
Oral temperature	96.8°F to 99.3°F (36.0°C to 37.4°C)	Conventional thermometer
Rectal temperature	97.3°F to 100.0°F (36.3°C to 37.8°C)	Conventional thermometer

Note:

- Temperatures measured with different thermometers should never be compared with one another.
- Tell your doctor what type of thermometer you used to take your temperature and on what part of the body. Also bear this in mind while you are monitoring yourself.

Holding the thermometer for too long in the hand before taking a measurement can cause the device to warm up. This means the measurement could be incorrect.

Influences On Body Temperature

- A person's individual metabolism
- Age
Body temperature is higher in babies and toddlers than in adults. Greater temperature fluctuations occur faster and more often in children. Normal body temperature decreases with age.
- Clothing

- Outside temperature
- Time of day
Body temperature is lower in the morning and increases throughout the day towards evening.
- Activities
Physical and, to a lesser extent, mental activities increase body temperature.

Note:

Taking the body temperature provides a current measurement of a person's temperature. If you are uncertain about interpreting the results or if the values are abnormal (e.g. fever), please consult your doctor. This also applies in the case of slight temperature changes if there are other symptoms of illness such as agitation, severe sweating, flushed skin, fast pulse rate, tendency to collapse, etc.

5. Initial Use

If provided, pull out the battery insulating strip at the battery compartment or remove the protective film from the battery and insert the battery with the correct polarity.

6. Switching On and Setting The Thermometer

The measuring sensor should be cleaned before and after each use, to eliminate dirt and fingerprints and ensure accuracy of your readings.

Briefly press the On/Off button

After a brief self-test and two short beeps the thermometer is ready for forehead measurement.

The device always starts up in forehead temperature mode

6.1 Setting The Basic Functions

This menu allows you to set the following functions individually, one after another.

Time format

- With the thermometer switched on, press and hold the **LIGHT/SET** button for 5 seconds.

The time format option flashes on the display (Fig. 1).

- Use the **MODE/MEM** button to set your preferred time format and confirm with the **LIGHT/SET** button.

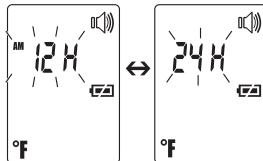
Fig. 1

Date

The year flashes in the display (Fig. 2).

- Use the **MODE/MEM** button to set the year and confirm with the **LIGHT/SET** button.

Month/day flashes in the display (Fig. 3).

- Use the **MODE/MEM** button to set the month and day, and confirm with the **LIGHT/SET** button.

i *In the 24h format, the date is displayed as day/month. In the 12h format, it is displayed as month/day.*

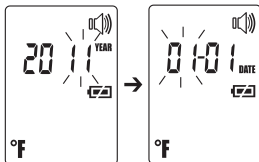
Fig. 2

Fig. 3

The time flashes in the display (Fig. 4).

- Use the **MODE/MEM** button to set the time and confirm with the **LIGHT/SET** button.
- i** In the 12h format, the time is displayed as AM/PM.

Fig. 4

The temperature measurement unit flashes in the display (Fig. 5).

You can set the device to display the temperature in degrees Fahrenheit (°F) or degrees Celsius (°C).

- To display the temperature in Celsius, select **°C** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To display the temperature in Fahrenheit, select **°F** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

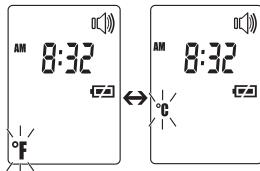
Fig. 5

The acoustic signal symbol flashes in the display (Fig. 6). You can activate/deactivate the acoustic signals (device activation, measurement in progress, measurement completion).

- To activate the acoustic signals, select **ON** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To deactivate the acoustic signals, select **OFF** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

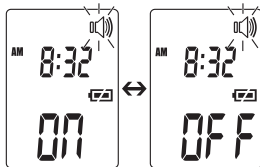
Fig. 6

The light symbol flashes in the display (Fig. 7).

You can activate or deactivate the automatic illuminated display (appears following forehead temperature measurement).

- To activate the automatic illuminated display, select  using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To deactivate the automatic illuminated display, select  using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

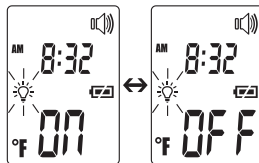
Fig. 7

You can also activate the illuminated display manually by briefly pressing the **LIGHT/SET** button.

The display lights up for 5 seconds.

- ⓘ The automatic and manual illuminated display cannot be used while measurement is in progress.

7. Forehead Measurement

Briefly press the On/Off button .

After a brief self-test and two short beeps the device is ready for forehead temperature measurement. The device is in forehead temperature mode. This is indicated by the  symbol.

Note:

Remember that

- physical activity, increased perspiration on the forehead, taking vasoconstrictive medication and skin irritations can distort the result.
- the forehead must be free from perspiration and cosmetics.
- before taking a reading, please remain indoors at room temperature for 5 minutes avoiding exercise or taking a bath for 30 min prior reading.
- in case of doubt, please measure several times and take the highest measurement value.

Hold the thermometer 0.78 – 1.18 in (2 – 3 cm) from the indicated point. Press the Start-measurement button and move the thermometer from side to side over the forehead area (Fig. 8). During measurement you will hear short beeps (only if the acoustic signal is activated), which signal that the thermometer has found a new highest measurement. End of measurement is signaled by a long beep. Release the Start-measurement button. You can now read the measurement value.

Fig. 8

Note:

Only one single measurement is taken at a single spot in the middle of the forehead.

The display also shows the symbols ☹ or ☺ depending on the measurement:

Symbol ☹ indicates that the body temperature is equal to or higher than 38°C, whereas symbol ☺ indicates a measured value below 100.4 °F (38°C).

If the acoustic signal is switched on, three beeps will sound once the measurement is complete if the temperature is equal to or higher than 100.4 °F (38°C).

The device is ready to take another measurement when two short beeps sound and the forehead symbol ☹ stops flashing.

The measurement is automatically saved with the date/time and classification ☹ / ☺.

Note:

Holding the thermometer in one place too long may cause a higher ambient temperature reading which could make the body temperature reading lower than usual.

7.1 Displaying Saved Measurements

The device only stores measurements in forehead temperature mode ☹. The device automatically stores the values from the last 60 measurements.

When 60 storage places are exceeded the oldest value is deleted.

The memory can be called up as follows:

- With the thermometer switched on, press and hold the **MODE/MEM** button for 5 seconds. The most recent measurement is displayed.
- With each subsequent pressing of the **MODE/MEM** button, first the storage space number is displayed followed by the measurement upon release.
- In the upper line, the time and date are displayed alternately.

8. Measuring Object Temperature /Room Temperature

Object temperature mode

If you want to measure an object temperature with this thermometer, switch to object temperature mode.

- With the thermometer switched on, briefly press the **MODE/MEM** button. The device switches to the object temperature mode .
- Hold the thermometer 0.78 – 1.18 in from the measuring point. Briefly press the **Start-measurement** button and read the temperature in the display (Fig. 9).

Measurements taken in object temperature mode are not stored.

Room temperature mode

If you want to use the thermometer to measure the room temperature you need to switch to room temperature mode.

- With the thermometer switched on, briefly press the **MODE/MEM** button twice.

The device switches to the room temperature mode .

- The room temperature is immediately displayed (Fig. 10).

Measurements taken in room temperature mode are not stored.

Fig. 9

Fig. 10


9. Troubleshooting Guide

Error message	Problem	Solution
Er 1	Measurement during self-test, device not yet ready for measurement.	Wait until the forehead symbol stops flashing.
Er 3	Room temperature below 50°F, $>104^{\circ}\text{F}$ (<math><10^{\circ}\text{C}</math> or over 40°C).	Room temperature must be between 50°F and 104°F (10°C and 40°C).
Hi	(1) Forehead temperature mode: The temperature recorded is higher than 108°F (42.2°C). (2) Object temperature mode: The temperature recorded is higher than 176°F (80°C).	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services.
Lo	(1) Forehead temperature mode: The temperature recorded is lower than 93.2°F (34°C). (2) Object temperature mode: The temperature recorded is lower than -7.6°F (-22°C).	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services.
	The batteries are empty.	Replace the batteries.

10. Changing batteries

The device requires two AAA (LR03) batteries.

- Open the battery compartment.
Use a pointed object to press down on the battery compartment lock and at the same time slide the battery compartment downwards.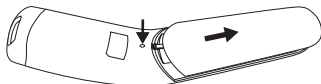

- Take the used batteries out of the battery compartment.
- Insert new batteries.
Make sure that the batteries are inserted observing the correct polarity.
- Close the battery compartment.

11. Care, Maintenance and Disposal

NOTICE:

- Do not subject the device to mechanical impacts.
 - When not in use, store the thermometer in the provided storage case.
 - Do not expose the device to liquids. The device is not waterproof. Avoid all direct contact with water or other liquids.
 - Do not expose the device to direct sunlight.
 - Clean the measuring sensor before each use.
 - Do not touch the measuring sensor with fingers.
-  • The measuring sensor is the most sensitive part of the thermometer. Take great care of the measuring sensor when cleaning the device.
- Do not use any harsh cleaning products.
 - Always observe all safety notes for user and device.
- Safety notes ▷ Page 3 – 6.

Clean the measuring sensor after each use. Use a clean cloth or cotton bud that can be moistened with disinfectant or 70 % alcohol.

To clean the entire device, please use a soft cloth slightly moistened with a mild soapy solution. Under no circumstances may liquid enter the device.

Do not use the device again until it is completely dry.

Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance to local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

12. Technical Specifications

If the device is used other than in accordance with the specifications, perfect functioning cannot be guaranteed!

We reserve the right to make technical changes to improve and develop the product.

Type	FT90/1
Measurement method	Non-contact infrared measurement
Basic functions	Forehead temperature measurement Object temperature measurement Room temperature measurement
Measurement units	Fahrenheit (°F) and Celsius (°C)
Operating conditions	50°F to 104°F (10°C to 40°C) with a relative humidity of $\leq 95\%$
Storage and Transport Conditions	-4°F to 122°F (-20°C to 50°C) with a relative humidity of $\leq 85\%$
Measurement distance	0.78 to 1.18 in (20 - 30 mm) from the measuring point
Measurement range and accuracy of forehead temperature measurement	Forehead temperature measurement 93.2°F to 108°F (34°C to 42.2°C) Measurement accuracy 96.8°F to 102°F: $\pm 0.4^\circ\text{F}$ (36°C to 39°C: $\pm 0.2^\circ\text{C}$) In the remaining measurement range $\pm 0.5^\circ\text{F}$ ($\pm 0.3^\circ\text{C}$)
Clinical repeat precision	0.41°F (0.23°C)
Measurement range and accuracy Object temperature measurement	Object temperature measurement -7.6°F to 176°F (-22°C to 80°C) Measurement accuracy $\pm 4\%$ or $\pm 4^\circ\text{F}$ ($\pm 2^\circ\text{C}$)
Memory function Body temperature measurement	Automatically stores the last 60 measurements.
Acoustic signal	With device activation, during measurement, when measurement is complete (can be set).

Display	LCD display
Energy saving functions	Device automatically switches off after 1 minute.
Dimensions Width x depth x height	aprox. 7.4 x 1.9 x 1.1 in (aprox. 188 x 47.6 x 29 mm)
Weight	2.9 oz (82 g) (without batteries)
2 x AAA (LR03) batteries	Batteries last for approx. 3000 measurements Activated functions such as acoustic signal or illuminated display also reduce the battery life.

These specifications are subject to change without notice for purpose of improvement. The serial number can be found on the unit itself or in the battery compartment.

This device meets the requirements of the EU Directive 93/42/EC concerning medical devices, as well as those of the Medizinproduktegesetz (German Medical Devices Act), the ASTM E 1965-98 and the European Standard EN 60601-1-2, and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.

13. FCC Compliance Information

Non-contact Forehead Thermometer FT 90

Responsible Party – U.S. Contact Information:

Beurer North America LP

1 Oakwood Boulevard, Suite 255

Hollywood, FL 33020

United States

1-800-536-0366

info@beurer.com

FCC Compliance Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

14. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer Non-contact Forehead Thermometer, model FT 90, excluding the battery, is warranted to be free from defects in materials and workmanship for the life of the product under

normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Non-contact Forehead Thermometer, model FT 90, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at **1-800-536-0366** or at **info@beurer.com** to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn battery; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed,

manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer. This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

Distributed by:
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020, USA

Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.

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Contracted Manufacturer: AViTA (WuJiang) Co. LTD